

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005956

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 650 psi Regulator: 900 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3060 Lot #: 202201A Exp. Date: 4/24/24 1/11/24
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3064 Lot #: 202012A Exp. Date: 12/8/22

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5114 Lot #: 1CS6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 02621080A1 Cyl #: 031 Exp. Date: 3/5/2023

Test 1 <u>0.076</u>	Test 4 <u>0.076</u>	Test 7 <u>0.077</u>
Test 2 <u>0.076</u>	Test 5 <u>0.077</u>	Test 8 <u>0.076</u>
Test 3 <u>0.076</u>	Test 6 <u>0.076</u>	Test 9 <u>0.077</u>
Average <u>0.076</u>		

C. Remarks/Maintenance: Annual inspection.

Replaced breath tube o-ring. Replaced external battery.

Instrument is acceptable to be used in the field.

Janelle Perkschiller

Breath Analyst Signature

Reviewed by

6/16/22

Date

062322

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005956
Location = TOXL 8164.14.00 09/16
06/16/2022 11:32

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Wet
Standard Value? 0.020
Standard Solution #? 202201A
Sim Serial #? MP3060
Oper No? 132128

Flow Cal. Date: 06/21/2013
Slope 706
Intercept -720901

IR Calibration Date: 04/02/2015
 3um 9um

0th Coef(*100): -23934 -22166
1st Coef(*100): 279293 133410
2nd Coef(*100): 3061 1559
H2O adj(mg/l*10k): 483 423

***** Printer Test End *****



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005956
Location = TOXL 8164.14.00 09/16
06/16/2022 11:33

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:34
02 Std. Sol.	0.017	11:40
03 Room Air	0.000	11:41
04 Std. Sol.	0.018	11:42
05 Room Air	0.000	11:42
06 Std. Sol.	0.018	11:43
07 Room Air	0.000	11:43

08 Sim Temp = 34.0°C

Simul Ser No = MP3060
Std Sol No = 202201A
County = 08 Oper No. = 132128


Operator Signature
JANELLE PORTSCHELLER

Remarks: *Low AC check 0.020*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005956
Location = TOXL 8164.14.00 09/16
06/16/2022 11:53

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:54
02 Std. Sol.	0.297	11:56
03 Room Air	0.000	11:56
04 Std. Sol.	0.299	11:57
05 Room Air	0.000	11:58
06 Std. Sol.	0.300	11:58
07 Room Air	0.000	11:59

08 Sim Temp = 34.0°C

Simul Ser No = MP3064

Std Sol No = 202012A

County = 08

Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *High AC check 0.300*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005956
Location = TOXL 8164.14.00 09/16
06/16/2022 12:08

Test	AC	Time
01 Room Air	0.000	12:08
02 *Subject Test	INT*	12:09
03 Room Air	0.000	12:10

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT CK
Dr. Lic. = ND/DIS821456
Lot No = 02621080A1
Cyl No = 31
Expiration Date = 03/05/2023
County = 08 Oper No. = 132128

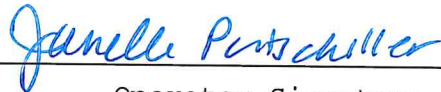

Operator Signature
JANELLE PORTSCHELLER

Remarks: ~~RFI check~~ ^{gwp}
Interferent Check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005956
Location = TOXL 8164.14.00 09/16
06/16/2022 12:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:15
02 Std. Gas	0.076	12:15
03 Room Air	0.000	12:16
04 Std. Gas	0.076	12:16
05 Room Air	0.000	12:17
06 Std. Gas	0.076	12:17
07 Room Air	0.000	12:18

Lot No = 02621080A1
Cyl No = 31
Exp Date = 03/05/2023
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #1-3*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005956
Location = TOXL 8164.14.00 09/16
06/16/2022 12:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:29
02 Std. Gas	0.076	12:29
03 Room Air	0.000	12:30
04 Std. Gas	0.077	12:30
05 Room Air	0.000	12:31
06 Std. Gas	0.076	12:31
07 Room Air	0.000	12:31

Lot No = 02621080A1
Cyl No = 31
Exp Date = 03/05/2023
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #4-4*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005956
Location = TOXL 8164.14.00 09/16
06/16/2022 12:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:35
02 Std. Gas	0.077	12:36
03 Room Air	0.000	12:36
04 Std. Gas	0.076	12:37
05 Room Air	0.000	12:37
06 Std. Gas	0.077	12:38
07 Room Air	0.000	12:38

Lot No = 02621080A1
Cyl No = 31
Exp Date = 03/05/2023
County = 08 Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #7-9*