

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-041004945

Location: ToxL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 653 psi Regulator: 650 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP6036 Lot #: 21340 Exp. Date: 8/24/23
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3068 Lot #: 202201F Exp. Date: 1/18/24

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3847 Lot #: 1CS 6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 02621080A1 Cyl #: 2 Exp. Date: 3/5/23

Test 1 <u>0.080</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

C. Remarks/Maintenance: Annual inspection

Replaced breath tube o-ring and simulator return o-ring.

Instrument is acceptable to be used in the field.

Janelle Pertschiller
Breath Analyst Signature

6/17/22
Date

[Signature]
Reviewed by

062322
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
06/17/2022 10:32

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 02621080A1
Standard Cyl #? 7
Standard Expiration? 03/05/2023
Oper No? 132128

Flow Cal. Date: 09/26/2012
Slope 622
Intercept -485836

IR Calibration Date: 04/22/2016
 3um 9um

0th Coef(*100): -23427 -26406
1st Coef(*100): 266989 144341
2nd Coef(*100): 2085 1078
H2O adj(mg/l*10k): 539 400

***** Printer Test End *****



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
06/17/2022 10:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:38
02 Std. Sol.	0.019	10:39
03 Room Air	0.000	10:40
04 Std. Sol.	0.018	10:41
05 Room Air	0.000	10:41
06 Std. Sol.	0.019	10:42
07 Room Air	0.000	10:42

08 Sim Temp = 34.0°C

Simul Ser No = MP6036

Std Sol No = 21340

County = 08

Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Low Ac check 0.020*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
06/17/2022 10:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:45
02 Std. Sol.	0.302	10:46
03 Room Air	0.000	10:46
04 Std. Sol.	0.302	10:47
05 Room Air	0.000	10:48
06 Std. Sol.	0.301	10:48
07 Room Air	0.000	10:49

08 Sim Temp = 34.0°C

Simul Ser No = MP3068
Std Sol No = 202201F
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *High AC check 0.300*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
06/17/2022 10:52

Test	AC	Time
01 Room Air	0.000	10:52
02 *Subject Test	INT*	10:54
03 Room Air	0.000	10:55

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT CK
Dr. Lic. = ND/DIS921456
Lot No = 02621080A1
Cyl No = 2
Expiration Date = 03/05/2023
County = 08 Oper No. = 132128

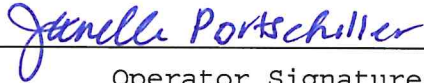

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Interferent check*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
06/17/2022 11:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:04
02 Std. Gas	0.080	11:04
03 Room Air	0.000	11:05
04 Std. Gas	0.079	11:05
05 Room Air	0.000	11:06
06 Std. Gas	0.080	11:06
07 Room Air	0.000	11:07

Lot No = 02621080A1
Cyl No = 2
Exp Date = 03/05/2023
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration check #1-3*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
06/17/2022 11:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:23
02 Std. Gas	0.079	11:23
03 Room Air	0.000	11:24
04 Std. Gas	0.080	11:24
05 Room Air	0.000	11:25
06 Std. Gas	0.079	11:25
07 Room Air	0.000	11:26

Lot No = 02621080A1
Cyl No = 2
Exp Date = 03/05/2023
County = 08 Oper No. = 132128

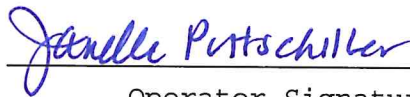

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #7-9*