

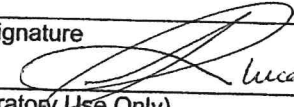
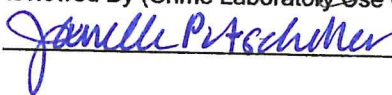


ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) Lucas Kuntz		
Location Wishek Police Department		Intoxilyzer® Serial Number 80-005952
Gas Lot Number 02621080A1	Gas Cylinder Number 21	Gas Expiration Date 03/05/2023

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.0 8 0 AC
 - B. 0.0 7 5 AC
 - C. 0.0 8 0 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature  Lucas Kuntz 1664	Date 06/18/2022
Reviewed By (Crime Laboratory Use Only) 	Date 06/24/22

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = WISH 8164.14.00 09/16
06/18/2022 14:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:55
02 Std. Gas	0.080	14:55
03 Room Air	0.000	14:56
04 Std. Gas	0.078	14:56
05 Room Air	0.000	14:57
06 Std. Gas	0.080	14:57
07 Room Air	0.000	14:58

Lot No = 02621080A1
Cyl No = 21
Exp Date = 03/05/2023
County = 26 Oper No. = 131099


Operator Signature
LUCAS KUNTZ

Remarks: Install ACA June
 new instrument
 Form 106-I8000