



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Troy N Nielsen</i>		
Location <i>Fargo Police PD3</i>	Intoxilyzer® Serial Number <i>90-004199</i>	
Gas Lot Number <i>02621080A1</i>	Gas Cylinder Number <i>13</i>	Gas Expiration Date <i>3-5-23</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.077* AC
 B. *0.078* AC
 C. *0.077* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>Troy Nielsen</i>	Date <i>6-24-22</i>
Reviewed By (Crime Laboratory Use Only) <i>Janelle Putschiller</i>	Date <i>06/30/2022</i>

Note: I called Troy Nielsen on 27th 21 July 22 to change the location code on the instrument. The location code was changed from FG PD to FAPD. Troy verified no subject tests have been run on the instrument since the installation. jpp 21 July 22

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = FGPD 8164.14.00 09/16
06/24/2022 09:24

*Location code
FAPD 21 July 22 9u*

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:24
02 Std. Gas	0.077	09:25
03 Room Air	0.000	09:25
04 Std. Gas	0.078	09:26
05 Room Air	0.000	09:26
06 Std. Gas	0.077	09:27
07 Room Air	0.000	09:27

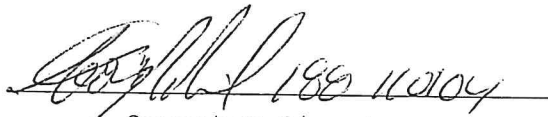
Lot No = 02621080A1

Cyl No = 13

Exp Date = 03/05/2023

County = 09

Oper No. = 110104


Operator Signature
TROY NIELSEN

Remarks: