

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00497

Location: TOXL

A. General Setup and Checks:

1. ☒ Diagnostic Tests Pass and Instrument in Ready Mode
2. ☒ Breath tube heated
3. ☒ Date, time and location code (Level 2,E). Re-set if necessary.
4. ☒ Print test (Level 1,P). Sign and attach test record.
5. ☒ Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 995 psi Regulator: 1600 psi

B. Tests (Sign and attach test records):

☒ Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. ☒ Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3070 Lot #: 22CP41184 Exp. Date: 3/1/24
- b. ☒ High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3065 Lot #: 202103E Exp. Date: 3/24/23

☒ Configure dry gas standard for the following tests (Level 1,S).

2. ☒ Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3847 Lot #: 16S6 Exp. Date: NA
☒ Display reads "Interferent Detect"
3. ☒ RFI Check (CMS Mode) ☒ Display reads "RFI Detect"
4. ☒ Dry Calibration Check (Level 1,C): Known Value 0.08 AC
Gas Cyl Lot #: 34921080A1 Cyl #: 44 Exp. Date: 2/5/24

Test 1 <u>0.080</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

C. Remarks/Maintenance: Annual inspection.

Replaced breath tube o-ring + simulator return o-ring.
Per prior annual inspections, this instrument has a flow sensor
issue and is for classroom use only.

NO. ☐ Instrument is acceptable to be used in the field. NO

Jamille Portschiller
Breath Analyst Signature

06/27/2022
Date

[Signature]
Reviewed by

062922
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/27/2022 15:08

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 24119080A1
Standard Cyl #? 5
Standard Expiration? 11/05/2021
Oper No? 132128

Flow Cal. Date: 06/26/2019
Slope 567
Intercept 293062

IR Calibration Date: 09/19/2016
 3um 9um

0th Coef(*100):	-15020	-25720
1st Coef(*100):	274116	137966
2nd Coef(*100):	1298	699
H2O adj(mg/l*10k):	226	399

***** Printer Test End *****

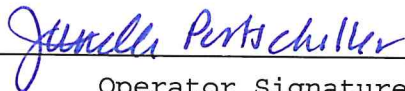

Operator Signature
JANELLE PORTSCHELLER

Remarks: Low Ac che 2nd Print test

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/27/2022 15:14

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:15
02 Std. Sol.	0.019	15:15
03 Room Air	0.000	15:16
04 Std. Sol.	0.019	15:17
05 Room Air	0.000	15:17
06 Std. Sol.	0.020	15:18
07 Room Air	0.000	15:18

08 Sim Temp = 34.0°C

Simul Ser No = MP3070
Std Sol No = 22CP41184
County = 08 Oper No. = 132128

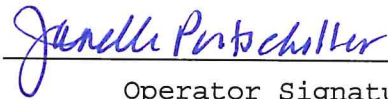

Operator Signature
JANELLE PORTSCHELLER

Remarks:

Low Ac check 0.020

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/27/2022 15:21

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:21
02 Std. Sol.	0.389	15:23
03 Room Air	0.000	15:23
04 Std. Sol.	0.389	15:24
05 Room Air	0.000	15:24
06 Std. Sol.	0.390	15:25
07 Room Air	0.000	15:26


08 Sim Temp = 34.0°C

Simul Ser No = MP3065

Std Sol No = 202103E

County = 08

Oper No. = 132128

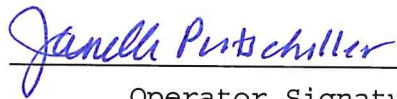

Operator Signature
JANELLE PORTSCHELLER

Remarks: *High Ac check 0.400*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/27/2022 15:35

Test	AC	Time
01 Room Air	0.000	15:36
02 *Subject Test	INT*	15:37
03 Room Air	0.000	15:37

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT CK
Dr. Lic. = ND/DIS821456
Lot No = 34921080A1
Cyl No = 44
Expiration Date = 02/05/2024
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Interferent check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/27/2022 15:40

Test	AC	Time
01 Diagnostic	OK	15:40
02 Room Air	0.000	15:41
03 *Subject Test	RFI*	15:42
04 Room Air	0.000	15:42

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHK
Dr. Lic. = ND/DIS821456
Lot No = 34921080A1
Cyl No = 44
Expiration Date = 02/05/2024
County = 08 Oper No. = 132128

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
JANELLE PORTSCHELLER

Remarks: *RFI check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/27/2022 15:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:47
02 Std. Gas	0.080	15:48
03 Room Air	0.000	15:48
04 Std. Gas	0.079	15:49
05 Room Air	0.000	15:49
06 Std. Gas	0.079	15:49
07 Room Air	0.000	15:50

Lot No = 34921080A1

Cyl No = 44

Exp Date = 02/05/2024

County = 08

Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Low^{9m} Dry Calibration Check #1-3*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/27/2022 15:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:58
02 Std. Gas	0.079	15:59
03 Room Air	0.000	15:59
04 Std. Gas	0.079	15:59
05 Room Air	0.000	16:00
06 Std. Gas	0.079	16:00
07 Room Air	0.000	16:01


Lot No = 34921080A1

Cyl No = 44

Exp Date = 02/05/2024

County = 08

Oper No. = 132128

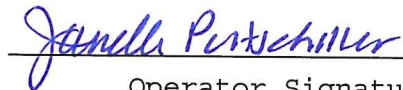

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration check #4-6*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/27/2022 16:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:30
02 Std. Gas	0.079	16:30
03 Room Air	0.000	16:31
04 Std. Gas	0.079	16:31
05 Room Air	0.000	16:32
06 Std. Gas	0.079	16:32
07 Room Air	0.000	16:33


Lot No = 34921080A1

Cyl No = 44

Exp Date = 02/05/2024

County = 08

Oper No. = 132128

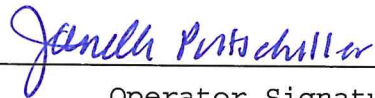

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check # 7-9*

Form 106-I8000