

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003057 Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 135 psi Regulator: 150 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3070 Lot #: 220P41184 Exp. Date: 3/1/24
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3065 Lot #: 202103E Exp. Date: 3/24/23

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3847 Lot #: 1CS 6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 02621080A1 Cyl #: 16 Exp. Date: 3/5/23

Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: Annual inspection.

Replace breath tube + simulator return v-rings.
Per prior annual inspections, this instrument has a flow sensor
ISSUE and is not acceptable for field use. Classroom use only.

No The instrument is not acceptable for field use.
 Instrument is acceptable to be used in the field.

Janelle Perkschiller
Breath Analyst Signature

06/27/22
Date

Reviewed by _____

062922
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
06/27/2022 10:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:53
02 Std. Sol.	0.018	10:54
03 Room Air	0.000	10:54
04 Std. Sol.	0.019	10:55
05 Room Air	0.000	10:55
06 Std. Sol.	0.019	10:56
07 Room Air	0.000	10:57


08 Sim Temp = 34.0°C

Simul Ser No = MP3070

Std Sol No = 22CP41184

County = 08

Oper No. = 132128

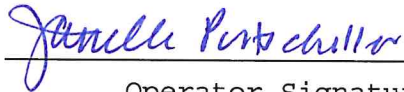

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Low Ac check 0.020*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
06/27/2022 11:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:59
02 Std. Gas	0.079	11:59
03 Room Air	0.000	12:00
04 Std. Gas	0.080	12:00
05 Room Air	0.000	12:01
06 Std. Gas	0.079	12:01
07 Room Air	0.000	12:02


Lot No = 02621080A1

Cyl No = 16

Exp Date = 03/05/2023

County = 08

Oper No. = 132128

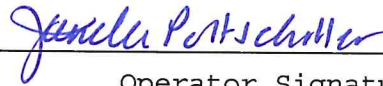

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #1-3*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
06/27/2022 12:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:03
02 Std. Gas	0.080	12:03
03 Room Air	0.000	12:03
04 Std. Gas	0.081	12:04
05 Room Air	0.000	12:04
06 Std. Gas	0.079	12:05
07 Room Air	0.000	12:05

Lot No = 02621080A1

Cyl No = 16

Exp Date = 03/05/2023

County = 08

Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks:

Dry Calibration Check # 4-b

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
06/27/2022 12:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:10
02 Std. Gas	0.080	12:10
03 Room Air	0.000	12:10
04 Std. Gas	0.079	12:11
05 Room Air	0.000	12:11
06 Std. Gas	0.080	12:12
07 Room Air	0.000	12:12

Lot No = 02621080A1

Cyl No = 16

Exp Date = 03/05/2023

County = 08

Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check # 7-9*