

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003066

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 784 psi Regulator: 750 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
    - a.  Low AC (Level 1,C): Known Value  $\leq$  0.03 AC: 0.020 AC  
Sim. Ser #: MP3060 Lot #: 202201A Exp. Date: 1/11/24
    - b.  High AC (Level 1,C): Known Value  $\geq$  0.25 AC: 0.300 AC  
Sim. Ser #: MP3064 Lot #: 202012A Exp. Date: 12/8/22

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7345 Lot #: 1CS7 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.08 AC  
Gas Cyl Lot #: 34921080A1 Cyl #: 49 Exp. Date: 2/5/24

Test 1 <u>0.078</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

C. Remarks/Maintenance: Annual inspection  
Replaced breath tube + simulator return o-rings.

Instrument is acceptable to be used in the field.

Janelle Pertschiller  
Breath Analyst Signature

06/29/2022  
Date

[Signature]  
Reviewed by

063022  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003066  
Location = TOXL                      8164.14.00 09/16  
06/29/2022                              09:44

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                      No  
Disable On Memfull?                Yes  
# of Print Copies?                    1  
Select Std (D/W/I)?                Dry  
Standard Value?                      0.080  
Standard Lot #?                      24119080A1  
Standard Cyl #?                      33  
Standard Expiration?                11/05/2021  
Oper No?                              132128

Flow Cal. Date:                      07/29/2010  
Slope                                806  
Intercept                            -895033

IR Calibration Date:                07/29/2010  
    3um                                    9um

-----  
0th Coef(\*100):                    -14075                                -5808  
1st Coef(\*100):                    272113                                138204  
2nd Coef(\*100):                    1637                                    986  
H2O adj(mg/l\*10k):                289                                    148

\*\*\*\*\* Printer Test End \*\*\*\*\*

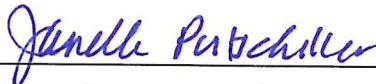  
\_\_\_\_\_  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/29/2022      09:48

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:49
02 Std. Sol.	0.020	09:49
03 Room Air	0.000	09:50
04 Std. Sol.	0.020	09:51
05 Room Air	0.000	09:51
06 Std. Sol.	0.020	09:52
07 Room Air	0.000	09:52

08 Sim Temp = 34.0°C

Simul Ser No = MP3060

Std Sol No = 202201A

County = 08

Oper No. = 132128



Operator Signature  
JANELLE PORTSCHELLER

Remarks:

*Low AC check 0.020*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/29/2022      09:54

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:55
02 Std. Sol.	0.297	09:56
03 Room Air	0.000	09:56
04 Std. Sol.	0.297	09:57
05 Room Air	0.000	09:58
06 Std. Sol.	0.297	09:58
07 Room Air	0.000	09:59


08 Sim Temp = 34.0°C

Simul Ser No = MP3064

Std Sol No = 202012A

County = 08

Oper No. = 132128

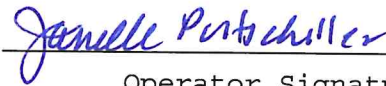

Operator Signature  
JANELLE PORTSCHELLER

Remarks: *High AC check 0.300*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/29/2022      10:02

Test	AC	Time
01 Room Air	0.000	10:02
02 *Subject Test	INT*	10:04
03 Room Air	0.000	10:05

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT CK  
Dr. Lic. = ND/DIS821456  
Lot No = 34921080A1  
Cyl No = 49  
Expiration Date = 02/05/2024  
County = 08      Oper No. = 132128



Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Interferent check*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003066  
Location = TOXL                 8164.14.00 09/16  
06/29/2022                        10:24

Test	AC	Time
01 Diagnostic	OK	10:24
02 Room Air	0.000	10:25
03 *Subject Test	RFI*	10:27
04 Room Air	0.000	10:27

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                     Weight = 150  
Test = DUI                             Cit = RFI CHK  
Dr. Lic. = ND/DIS821456  
Lot No = 34921080A1  
Cyl No = 49  
Expiration Date = 02/05/2024  
County = 08                            Oper No. = 132128

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

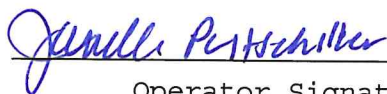

Operator Signature  
JANELLE PORTSCHELLER

Remarks: *RFI check*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/29/2022      10:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:29
02 Std. Gas	0.078	10:30
03 Room Air	0.000	10:30
04 Std. Gas	0.079	10:30
05 Room Air	0.000	10:31
06 Std. Gas	0.079	10:31
07 Room Air	0.000	10:32

Lot No = 34921080A1  
Cyl No = 49  
Exp Date = 02/05/2024  
County = 08      Oper No. = 132128

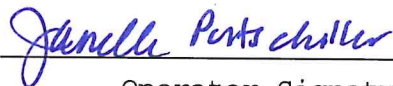  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #1-3*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/29/2022      10:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:36
02 Std. Gas	0.079	10:36
03 Room Air	0.000	10:37
04 Std. Gas	0.079	10:37
05 Room Air	0.000	10:38
06 Std. Gas	0.079	10:38
07 Room Air	0.000	10:39

Lot No = 34921080A1  
Cyl No = 49  
Exp Date = 02/05/2024  
County = 08      Oper No. = 132128

*Janelle Portscheller*  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #4-6*




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/29/2022      10:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:41
02 Std. Gas	0.080	10:41
03 Room Air	0.000	10:41
04 Std. Gas	0.079	10:42
05 Room Air	0.000	10:42
06 Std. Gas	0.079	10:43
07 Room Air	0.000	10:43

Lot No = 34921080A1  
Cyl No = 49  
Exp Date = 02/05/2024  
County = 08      Oper No. = 132128

  
\_\_\_\_\_  
Operator Signature  
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check # 7-9*