

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00 4194

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 750 psi Regulator: 750 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3060 Lot #: 202201A Exp. Date: 1/11/24
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3064 Lot #: 202012A Exp. Date: 12/8/22

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7345 Lot #: 1C57 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.08 AC
Gas Cyl Lot #: 34921080A1 Cyl #: 49 Exp. Date: 2/5/24

Test 1 <u>0.078</u>	Test 4 <u>0.079</u>	Test 7 <u>0.080</u>
Test 2 <u>0.078</u>	Test 5 <u>0.078</u>	Test 8 <u>0.078</u>
Test 3 <u>0.080</u>	Test 6 <u>0.079</u>	Test 9 <u>0.078</u>
Average <u>0.079</u>		

C. Remarks/Maintenance: Annual inspection.
Replaced breath tube + simulator return o-rings.

Instrument is acceptable to be used in the field.

Janella Purkschiller
Breath Analyst Signature

6/29/2022
Date

[Signature]
Reviewed by

063022
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.14.00 09/16
06/29/2022 13:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:02
02 Std. Sol.	0.018	13:03
03 Room Air	0.000	13:03
04 Std. Sol.	0.018	13:04
05 Room Air	0.000	13:05
06 Std. Sol.	0.018	13:05
07 Room Air	0.000	13:06

08 Sim Temp = 34.0°C

Simul Ser No = MP3060
Std Sol No = 202201A
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Low Ac Check 0.020*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.14.00 09/16
06/29/2022 13:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:26
02 Std. Gas	0.078	13:26
03 Room Air	0.000	13:27
04 Std. Gas	0.078	13:27
05 Room Air	0.000	13:28
06 Std. Gas	0.080	13:28
07 Room Air	0.000	13:29

Lot No = 34921080A1

Cyl No = 49

Exp Date = 02/05/2024

County = 08

Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #1-3*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.14.00 09/16
06/29/2022 13:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:33
02 Std. Gas	0.079	13:33
03 Room Air	0.000	13:34
04 Std. Gas	0.078	13:34
05 Room Air	0.000	13:35
06 Std. Gas	0.079	13:35
07 Room Air	0.000	13:36

Lot No = 34921080A1
Cyl No = 49
Exp Date = 02/05/2024
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #1-3 4-6*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = TOXL 8164.14.00 09/16
06/29/2022 13:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:37
02 Std. Gas	0.080	13:37
03 Room Air	0.000	13:38
04 Std. Gas	0.078	13:38
05 Room Air	0.000	13:39
06 Std. Gas	0.078	13:39
07 Room Air	0.000	13:40

Lot No = 34921080A1
Cyl No = 49
Exp Date = 02/05/2024
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: Dry Calibration check # + ← 3^{pm} 7-9