



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-005358	Instrument Location Burleigh Morton Detention Center
Reason for Install/Repair	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Handwritten Signature</i>	Date 9-30-22
---	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Handwritten Signature</i>	Certified Date 06 Oct 22
---	------------------------------------



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Harley Sterna</i>		
Location <i>Burleigh Morton Detention Center</i>	Intoxilyzer® Serial Number <i>80-005358</i>	
Gas Lot Number <i>02621080A1</i>	Gas Cylinder Number <i>9</i>	Gas Expiration Date <i>3-5-23</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.081* AC
 - B. *0.082* AC
 - C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Harley Sterna</i>	Date <i>9-30-22</i>
Reviewed By (Crime Laboratory Use Only) <i>Chris Dahl</i>	Date <i>06 Oct 22</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = BMDC 8164.14.00 09/16
09/30/2022 15:05

Test	AC	Time
01 Room Air	RFI*	15:05
02 Room Air	0.000	15:06

*Invalid Test
Inhibited - RFI

Sub Name = DOE, JOHN J
Sub DOB = 09/30/2000
Sub Sex = Male Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 02621080A1
Cyl No = 9
Expiration Date = 03/05/2023
County = 08 Oper No. = 132522


Operator Signature
KARLEY STERNA

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = BMDC 8164.14.00 09/16
09/30/2022 15:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:01
02 Std. Gas	0.081	15:02
03 Room Air	0.000	15:02
04 Std. Gas	0.082	15:03
05 Room Air	0.000	15:03
06 Std. Gas	0.081	15:04
07 Room Air	0.000	15:04

Lot No = 02621080A1
Cyl No = 9
Exp Date = 03/05/2023
County = 08 Oper No. = 132522


Operator Signature
KARLEY STERNA

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = BMDC 8164.14.00 09/16
09/30/2022 14:53

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 02621080A1
Standard Cyl #? 9
Standard Expiration? 03/05/2023
Oper No? 132522

Flow Cal. Date: 06/04/2019
Slope 570
Intercept 4079

IR Calibration Date: 06/04/2019

	3um	9um
0th Coef(*100):	-19471	-24652
1st Coef(*100):	273237	143582
2nd Coef(*100):	2965	1293
H2O adj (mg/l*10k):	273	374

***** Printer Test End *****


Operator Signature
KARLEY STERNA

Remarks: