



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-003061	Instrument Location Grand Forks County Corrections (GFSO)
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): Instrument exchange
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 11/9/2022
--	-------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date 14 Nov 2022
--	-------------------------------

Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000 SN 80-003061
 Location = GFSO 8164.14.00 09/16
 11/09/2022 10:12

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
 Start Test Sequence: DABACABA
 Display Prelim Rslt? Yes
 Display Third Digit? Yes
 Inhib Printer(Y/N)? No
 Display Volume? No
 Disable On Memfull? Yes
 # of Print Copies? 1
 Select Std (D/W/I)? Dry
 Standard Value? 0.080
 Standard Lot #? 26021080A1
 Standard Cyl #? 3
 Standard Expiration? 10/05/2023
 Oper No? 020836

Flow Cal. Date: 03/04/2011
 Slope 700
 Intercept -623726

IR Calibration Date: 04/10/2015

	3um	9um
0th Coef(*100):	-25722	-17879
1st Coef(*100):	266163	136703
2nd Coef(*100):	2302	1151
H2O adj(mg/l*10k):	412	294

***** Printer Test End *****


 Operator Signature
 DAVID STROMBERG

Remarks:

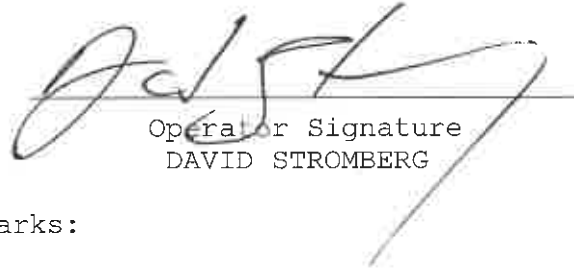
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = GFSO 8164.14.00 09/16
11/09/2022 10:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:14
02 Std. Gas	0.079	10:14
03 Room Air	0.000	10:15
04 Std. Gas	0.080	10:15
05 Room Air	0.000	10:15
06 Std. Gas	0.080	10:16
07 Room Air	0.000	10:16

Lot No = 26021080A1
Cyl No = 3
Exp Date = 10/05/2023
County = 18 Oper No. = 020836


Operator Signature
DAVID STROMBERG

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = GFSO 8164.14.00 09/16
11/09/2022 10:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	10:20
02 Room Air	0.000	10:20

*Invalid Test
Inhibited - RFI

Lot No = 26021080A1
Cyl No = 3
Exp Date = 10/05/2023
County = 18 Oper No. = 020836


Operator Signature
DAVID STROMBERG

Remarks:

Form 106-I8000