



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Michelle Romans</i>		
Location <i>Williston</i>	Intoxilyzer® Serial Number <i>80-004944</i>	
Gas Lot Number <i>26021080A1</i>	Gas Cylinder Number <i>13</i>	Gas Expiration Date <i>10/5/2023</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. *0.083* AC  
 B. *0.082* AC  
 C. *0.082* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:  
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).  
 B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>3-7-23</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>08Mar23</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004944  
Location = WILL      8164.14.00 09/16  
03/07/2023      14:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:12
02 Std. Gas	0.083	14:12
03 Room Air	0.000	14:13
04 Std. Gas	0.082	14:13
05 Room Air	0.000	14:14
06 Std. Gas	0.082	14:14
07 Room Air	0.000	14:15

Lot No = 26021080A1  
Cyl No = 13  
Exp Date = 10/05/2023  
County = 53      Oper No. = 130739

  
Operator Signature  
MICHELLE ROMANS

Remarks:

ACA Check

Form 106-I8000