



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (5-2011)

Serial Number <i>80-005360</i>	Instrument Location <i>ASHL</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): *NOT APPLICABLE*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i> <i>UCSD</i>	Date <i>7/3/2012</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i>	Date <i>11 July 2012</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *Charles E. Edin*      Certified Date: *23 Nov. 2016*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005360  
Location = ASHL                 8164.13.00 06/09  
07/03/2012                        19:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	19:54
02 Std. Gas	0.080	19:55
03 Room Air	0.000	19:55
04 Std. Gas	0.080	19:56
05 Room Air	0.000	19:56
06 Std. Gas	0.081	19:57
07 Room Air	0.000	19:57

Lot No = 23411080A1  
Cyl No = 48  
Exp Date = 10/01/2013  
County = 26                        Oper No. = 100631

  
\_\_\_\_\_  
Operator Signature  
GERARD ZLOTKOWSKI

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005360  
Location = ASHL            8164.13.00 06/09  
07/03/2012                            20:13

Test	AC	Time
01 Diagnostic	OK	20:15
02 Room Air	0.000	20:15
03 *Subject Test	RFI*	20:15
04 Room Air	0.000	20:16

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST TEST  
Sub DOB = 01/01/1970  
Sub Sex = Male                            Weight = 200  
Test = DUI    Cit = TEST  
Dr. Lic. = ND/TEST  
Lot No = 23411080A1  
Cyl No = 48  
Expiration Date = 10/01/2013  
County = 26                            Oper No. = 100631

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
GERARD ZLOTKOWSKI

Remarks:

Form 106-I8000