



NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION

INTOXILYZER® 8000 ANNUAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00 2647 Inspection Location: TOXL

A. Pre-Inspection

1. Items with Instrument:
 - Gas Cylinder Yes or No (If Yes, Lot # _____ Cyl. # _____)
 - Keys Yes or No
 - Power Cord Yes or No
2. Download Data
3. Upload Operator File
4. Current Location Code: TOXL
5. Battery Check
 - Was the external battery pack replaced? Yes or No
6. O-Rings
 - Replaced Simulator O-Ring Yes or No
 - Replaced Breath Tube O-Ring Yes or No

B. General Setup and Checks:

1. Diagnostics passed and instrument in "Ready" mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
Time Zone: CST or MDT (Time on test records will be in time zone circled).
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G).
Display: 331 psi Regulator: 350 psi
Display and Regulator ± 50 psi of each other Yes or No
Gas tank tare necessary? Yes or No
If Yes, display readings after tare (Level 3,M,C,G):
Display: _____ psi Regulator: _____ psi

C. Tests (Sign and attach test records):

1. Configure simulator for the following test (Level 1,S).
Wet Calibration Check - Low AC (Level 1,C)
Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP5318
Lot #: 202201A
Exp. Date: 11Jan24
 Results ± 0.005 of known AC

Toxicology Unit - Breath Alcohol Section	Qualtrax ID: 11698 Revision: 1
Intoxilyzer 8000 Annual Inspection	Status: Published Published: 03/24/2023
UNCONTROLLED WHEN PRINTED	Page 1 of 2

2. Configure simulator for the following test (Level 1,S).
 Wet Calibration Check - High AC (Level 1,C)
 Known Value \geq 0.25 AC: 0.300 AC
 Sim. Ser #: MP3067
 Lot #: 2022DIF
 Exp. Date: 18Jan24
 Results \pm 5% AC of known AC
3. Configure dry gas standard for the remaining tests (Level 1,S).
 Known Value : 0.080 AC
 Gas Cylinder Lot #: 2602108DA1
 Cylinder #: 24
 Exp. Date: 10/5/23
4. Interferent Check (Level 1,B)
 Known Value: 0.10 AC + 0.05% Acetone
 Sim. Ser #: DR7345
 Lot #: ICS 7
 Exp. Date: NIA
 Display reads "Interferent Detect"
5. RFI Check (CMS Mode)
 Display reads "RFI Detect"
6. Dry Calibration Check (Level 1,C)
 Test 1 0.080 Test 4 0.080 Test 7 0.080
 Test 2 0.080 Test 5 0.080 Test 8 0.080
 Test 3 0.080 Test 6 0.080 Test 9 0.080
 Average 0.080
 Results \pm 0.005 AC of known AC

D. Remarks/Maintenance: NIA

Instrument is acceptable to be used in the field. Yes or No
 If No, state reason(s) why: Fan motor does not turn on.
Classroom only instrument.

If Yes, change location code back to A.4.

[Signature]
 Inspector Signature

24 May 2023
 Date

Janelle Pertschiller
 Reviewer

25 May 2023
 Date

Toxicology Unit - Breath Alcohol Section	Qualtrax ID: 11698 Revision: 1
Intoxilyzer 8000 Annual Inspection	Status: Published Published: 03/24/2023
UNCONTROLLED WHEN PRINTED	Page 2 of 2

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = TOXL 8164.14.00 09/16
05/24/2023 14:08

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:09
02 Std. Sol.	0.022	14:09
03 Room Air	0.000	14:10
04 Std. Sol.	0.022	14:11
05 Room Air	0.000	14:11
06 Std. Sol.	0.022	14:12
07 Room Air	0.000	14:12

08 Sim Temp = 34.0°C

Simul Ser No = MP5318

Std Sol No = 202201A

County = 08

Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: *LOW Ac check- 0.020Ac*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = TOXL 8164.14.00 09/16
05/24/2023 14:14

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:14
02 Std. Sol.	0.303	14:15
03 Room Air	0.000	14:15
04 Std. Sol.	0.306	14:16
05 Room Air	0.000	14:17
06 Std. Sol.	0.306	14:17
07 Room Air	0.000	14:18

08 Sim Temp = 34.0°C

Simul Ser No = MP3067
Std Sol No = 202201F
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: High Ac Check- 0.300Ac

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = TOXL 8164.14.00 09/16
05/24/2023 14:19

Test	AC	Time
01 Room Air	0.000	14:20
02 *Subject Test	INT*	14:20
03 Room Air	0.000	14:21

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = OTH Cit = INTERFERENT CK
Dr. Lic. = ND/TES989643
Lot No = 26021080A1
Cyl No = 24
Expiration Date = 10/05/2023
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: *Interferent Check*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = TOXL 8164.14.00 09/16
05/24/2023 14:21

Test	AC	Time
01 Diagnostic	OK	14:22
02 Room Air	0.000	14:23
03 *Subject Test	RFI*	14:23
04 Room Air	0.000	14:23

*Invalid Test
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = OTH Cit = RFI CHECK
Dr. Lic. = ND/TES989643
Lot No = 26021080A1
Cyl No = 24
Expiration Date = 10/05/2023
County = 08 Oper No. = 133237

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
ANNA NAREHOOD

Remarks: *RFI check*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = TOXL 8164.14.00 09/16
05/24/2023 14:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:25
02 Std. Gas	0.080	14:25
03 Room Air	0.000	14:26
04 Std. Gas	0.080	14:26
05 Room Air	0.000	14:26
06 Std. Gas	0.080	14:27
07 Room Air	0.000	14:27

Lot No = 26021080A1
Cyl No = 24
Exp Date = 10/05/2023
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: *Dry cal check #1-3*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = TOXL 8164.14.00 09/16
05/24/2023 14:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:28
02 Std. Gas	0.080	14:28
03 Room Air	0.000	14:29
04 Std. Gas	0.080	14:29
05 Room Air	0.000	14:30
06 Std. Gas	0.080	14:30
07 Room Air	0.000	14:30

Lot No = 26021080A1
Cyl No = 24
Exp Date = 10/05/2023
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: *Dry Cal check #4-6*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002667
Location = TOXL 8164.14.00 09/16
05/24/2023 14:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:32
02 Std. Gas	0.080	14:32
03 Room Air	0.000	14:33
04 Std. Gas	0.081	14:33
05 Room Air	0.000	14:34
06 Std. Gas	0.080	14:34
07 Room Air	0.000	14:35

Lot No = 26021080A1
Cyl No = 24
Exp Date = 10/05/2023
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: *Dry cal check #7-9*

Form 106-I8000