



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-006505	Instrument Location Mohall - Renville County S.O.
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) Annual Inspection at Crime Laboratory	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): **Annual Inspection**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date 5/30/23
---	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>[Signature]</i>	Certified Date 30May2023
---	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006505
Location = MOHL 8164.14.00 09/16
05/30/2023 11:29

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer (Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 26021080A1
Standard Cyl #? 23
Standard Expiration? 10/05/2023
Oper No? 133141

Flow Cal. Date: 08/20/2015
Slope 671
Intercept -642269

IR Calibration Date:	08/20/2015
	3um 9um
0th Coef(*100):	-29025 -17051
1st Coef(*100):	252705 133345
2nd Coef(*100):	3840 1546
H2O adj(mg/l*10k):	817 461

***** Printer Test End *****



Operator Signature
ALEX COOK

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006505
Location = MOHL 8164.14.00 09/16
05/30/2023 11:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:35
02 Std. Gas	0.079	11:36
03 Room Air	0.000	11:36
04 Std. Gas	0.078	11:36
05 Room Air	0.000	11:37
06 Std. Gas	0.078	11:37
07 Room Air	0.000	11:38

Lot No = 26021080A1
Cyl No = 23
Exp Date = 10/05/2023
County = 38 Oper No. = 133141



Operator Signature
ALEX COOK

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006505
Location = MOHL 8164.14.00 09/16
05/30/2023 11:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	11:39
02 Room Air	0.000	11:39

*Invalid Test
Inhibited - RFI

Lot No = 26021080A1
Cyl No = 23
Exp Date = 10/05/2023
County = 38 Oper No. = 133141



Operator Signature
ALEX COOK

Remarks:

Form 106-I8000