



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Joshua Nelson</i>		
Location <i>Tioga PD</i>	Intoxilyzer® Serial Number <i>80-007098</i>	
Gas Lot Number <i>349Z1080A1</i>	Gas Cylinder Number <i>028</i>	Gas Expiration Date <i>02/05/2024</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.081* AC
 B. *0.082* AC
 C. *0.080* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>Joshua Nelson</i>	Date <i>03-15-2023</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>15 Mar 2023</i>

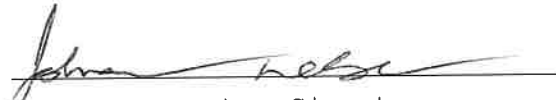
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007098
Location = TIOG 8164.16.00 09/18
03/15/2023 14:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:29
02 Std. Gas	0.081	14:29
03 Room Air	0.000	14:30
04 Std. Gas	0.082	14:30
05 Room Air	0.000	14:31
06 Std. Gas	0.080	14:31
07 Room Air	0.000	14:32

Lot No = 34921080A1
Cyl No = 28
Exp Date = 02/05/2024
County = 53 Oper No. = 130977


Operator Signature
JOSHUA NELSON

Remarks:

I certify this is a true
and accurate copy.

Form 106-I8000

X 