



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-006507	Instrument Location LAMO
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): none
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>A. Pelt</i>	Date 3-27-23
---------------------------------------------	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Am Muel</i>	Certified Date 29Mar2023
-----------------------------------------	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = ~~TOXL~~ LAMO 8164.14.00 09/16
03/27/2023 13:58

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 33321080A4
Standard Cyl #? 10
Standard Expiration? 01/05/2024
Oper No? 132828

Flow Cal. Date: 08/20/2015
Slope 659
Intercept -658315

IR Calibration Date: 08/20/2015
3um 9um

0th Coef(*100): -9452 -11722
1st Coef(*100): 258808 139588
2nd Coef(*100): 4476 1674
H2O adj(mg/l*10k): 499 376

***** Printer Test End *****



Operator Signature
ALEXIS PETERSON

Remarks:

Form 106-I8000

Confirmed correct
location code
29mar23
- AEN


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = ~~TOXL~~ LAMO 8164.14.00 09/16
03/27/2023 14:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:35
02 Std. Gas	0.081	14:36
03 Room Air	0.000	14:36
04 Std. Gas	0.080	14:36
05 Room Air	0.000	14:37
06 Std. Gas	0.080	14:37
07 Room Air	0.000	14:38

Lot No = 33321080A4
Cyl No = 10
Exp Date = 01/05/2024
County = 23 Oper No. = 132828


Operator Signature
ALEXIS PETERSON

Remarks:

Form 106-I8000

Confirmed correct
location code
29mar23
-aew

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006507
Location = ~~TOXIL~~ LAMO 8164.14.00 09/16
03/27/2023 14:39

Test	AC	Time
01 Room Air	RFI*	14:40
02 Room Air	RFI*	14:40

*Invalid Test
Inhibited - RFI



Operator Signature
ALEXIS PETERSON

Remarks: Setup

Form 106-I8000

Confirmed correct
location code
29mar23
-aew