



ETHANOL BREATH STANDARD CYLINDER REPORT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59282 (5-2011)

Chemical Test Operator Name (Print) EBERLE FRANK J		
Location Baumanco Sheriff's Office		Intoximeter® Serial Number 80-003071
Gas Lot Number 01	Gas Cylinder Number 014	Gas Expiration Date 10-5-2023

211543
 Check When Done

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S)
- 2. Perform an ACA Test (Level 1, Function C)
 Write in the ACA Test Results (Reported to 3 Digits, Ex: 0.081):
 A. 0.079 AC
 B. 0.078 AC
 C. 0.079 AC
- 3. Complete the Top Portion of the Intoximeter® Record (SFN50496, Form 120-G) and Place it by the Intoximeter® for Use
- 4. File Previous Intoximeter® Record (SFN50496, Form 120-G) at the Intoximeter® Location at the Agency
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Breath Standard Cylinder Report (SFN59282, Form 105-G)
 - B. ACA Test Record

Chemical Test Operator Signature Frank J Eberle	Date 4-21-2023
Reviewed By (Crime Laboratory Use Only) Am Will	Date 25Apr2023


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003071
Location = ~~BOWN~~ **Bowm** 8164.14.00 09/16
04/21/2023 **aen 25Apr23** 09:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:54
02 Std. Gas	0.079	09:54
03 Room Air	0.000	09:55
04 Std. Gas	0.078	09:55
05 Room Air	0.000	09:56
06 Std. Gas	0.079	09:56
07 Room Air	0.000	09:57

Lot No = 26021080A1
Cyl No = 014
Exp Date = 10/05/2023
County = 06 Oper No. = 130015



Operator Signature
FRANK EBERLE

Remarks:

Form 106-I8000