



NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION

INTOXILYZER® 8000 ANNUAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00 5945 Inspection Location: TOXL

A. Pre-Inspection

1. Items with Instrument:
 - Gas Cylinder Yes or No (If Yes, Lot # 34921080A1 Cyl. # 51)
 - Keys Yes or No
 - Power Cord Yes or No
2. Download Data
3. Upload Operator File
4. Current Location Code: BSPD
5. Battery Check
 - Was the external battery pack replaced? Yes or No
6. O-Rings
 - Replaced Simulator O-Ring Yes or No
 - Replaced Breath Tube O-Ring Yes or No

B. General Setup and Checks:

1. Diagnostics passed and instrument in "Ready" mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
Time Zone: CST or MDT (Time on test records will be in time zone circled).
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G).
Display: 417 psi Regulator: 400 psi
Display and Regulator ± 50 psi of each other Yes or No
Gas tank tare necessary? Yes or No
If Yes, display readings after tare (Level 3,M,C,G):
Display: _____ psi Regulator: _____ psi

C. Tests (Sign and attach test records):

1. Configure simulator for the following test (Level 1,S).
Wet Calibration Check - Low AC (Level 1,C)
Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP5318
Lot #: 202201A
Exp. Date: 11Jan24
 Results ± 0.005 of known AC

Toxicology Unit - Breath Alcohol Section	Qualtrax ID: 11698 Revision: 1
Intoxilyzer 8000 Annual Inspection	Status: Published Published: 03/24/2023
UNCONTROLLED WHEN PRINTED	Page 1 of 2

- 2. Configure simulator for the following test (Level 1,S).
 Wet Calibration Check - High AC (Level 1,C)
 Known Value \geq 0.25 AC: 0.300 AC
 Sim. Ser #: 20 MP6039
 Lot #: 202201F
 Exp. Date: 18Jan24
 Results \pm 5% AC of known AC
- 3. Configure dry gas standard for the remaining tests (Level 1,S).
 Known Value : 0.080 AC
 Gas Cylinder Lot #: 26021080A1
 Cylinder #: 16
 Exp. Date: 10/5/23
- 4. Interferent Check (Level 1,B)
 Known Value: 0.10 AC + 0.05% Acetone
 Sim. Ser #: DR3841
 Lot #: 1CS7
 Exp. Date: NIA
 Display reads "Interferent Detect"
- 5. RFI Check (CMS Mode)
 Display reads "RFI Detect"
- 6. Dry Calibration Check (Level 1,C)
 Test 1 0.079 Test 4 0.080 Test 7 0.080
 Test 2 0.079 Test 5 0.080 Test 8 0.080
 Test 3 0.079 Test 6 0.080 Test 9 0.080
 Average 0.079
 Results \pm 0.005 AC of known AC

D. Remarks/Maintenance: NIA

Instrument is acceptable to be used in the field. Yes or No

If No, state reason(s) why: _____

If Yes, change location code back to A.4.

Amuel
 Inspector Signature

25Apr2023
 Date

Janelle Pertschmer
 Reviewer

26Apr 2023
 Date

Toxicology Unit - Breath Alcohol Section	Qualtrax ID: 11698 Revision: 1
Intoxilyzer 8000 Annual Inspection	Status: Published Published: 03/24/2023
UNCONTROLLED WHEN PRINTED	Page 2 of 2

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
04/25/2023 11:06

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34921080A1
Standard Cyl #? 51
Standard Expiration? 02/05/2024
Oper No? 133237

Flow Cal. Date: 06/20/2013
Slope 637
Intercept -566407

IR Calibration Date: 05/22/2015

	3um	9um
0th Coef(*100):	-21213	-23093
1st Coef(*100):	279171	139292
2nd Coef(*100):	2898	1291
H2O adj(mg/l*10k):	332	319

***** Printer Test End *****

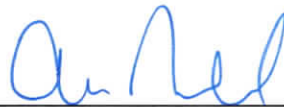

Operator Signature
ANNA NAREHOOD

Remarks: *print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
04/25/2023 11:10

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:11
02 Std. Sol.	0.020	11:12
03 Room Air	0.000	11:12
04 Std. Sol.	0.020	11:13
05 Room Air	0.000	11:13
06 Std. Sol.	0.019	11:14
07 Room Air	0.000	11:15


08 Sim Temp = 34.0°C

Simul Ser No = MP5318

Std Sol No = 202201A

County = 08

Oper No. = 133237

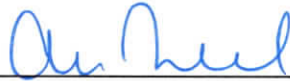

Operator Signature
ANNA NAREHOOD

Remarks: *LOW AC Check - 0.020AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
04/25/2023 11:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:16
02 Std. Sol.	0.306	11:17
03 Room Air	0.000	11:18
04 Std. Sol.	0.307	11:18
05 Room Air	0.000	11:19
06 Std. Sol.	0.306	11:20
07 Room Air	0.000	11:20

08 Sim Temp = 34.0°C

Simul Ser No = MP6039

Std Sol No = 202201F

County = 08

Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: High Ac Check - 0.300Ac

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
04/25/2023 11:21

Test	AC	Time
01 Room Air	0.000	11:22
02 *Subject Test	INT*	11:23
03 Room Air	0.000	11:24

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = OTH Cit = INTERFERENT CK
Dr. Lic. = ND/TES989643
Lot No = 26021080A1
Cyl No = 16
Expiration Date = 10/05/2023
County = 08 Oper No. = 133237

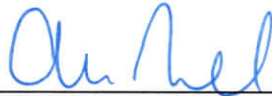

Operator Signature
ANNA NAREHOOD

Remarks: Interferent check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
04/25/2023 11:24

Test	AC	Time
01 Diagnostic	OK	11:25
02 Room Air	0.000	11:25
03 *Subject Test	RFI*	11:25
04 Room Air	0.000	11:26

*Invalid Test
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = OTH Cit = RFI CHECK
Dr. Lic. = ND/TES989643
Lot No = 26021080A1
Cyl No = 16
Expiration Date = 10/05/2023
County = 08 Oper No. = 133237

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
ANNA NAREHOOD

Remarks: RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
04/25/2023 11:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:27
02 Std. Gas	0.079	11:27
03 Room Air	0.000	11:28
04 Std. Gas	0.079	11:28
05 Room Air	0.000	11:29
06 Std. Gas	0.079	11:29
07 Room Air	0.000	11:30

Lot No = 26021080A1
Cyl No = 16
Exp Date = 10/05/2023
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: Dry cal check #1-3

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
04/25/2023 11:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:31
02 Std. Gas	0.080	11:31
03 Room Air	0.000	11:32
04 Std. Gas	0.080	11:32
05 Room Air	0.000	11:32
06 Std. Gas	0.080	11:33
07 Room Air	0.000	11:33

Lot No = 26021080A1
Cyl No = 16
Exp Date = 10/05/2023
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: Dry Cal check #4-6

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = TOXL 8164.14.00 09/16
04/25/2023 11:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:35
02 Std. Gas	0.080	11:35
03 Room Air	0.000	11:36
04 Std. Gas	0.080	11:36
05 Room Air	0.000	11:37
06 Std. Gas	0.080	11:37
07 Room Air	0.000	11:38

Lot No = 26021080A1
Cyl No = 16
Exp Date = 10/05/2023
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: Dry cal check #7-9

Form 106-I8000