



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <i>80-006679</i>	Instrument Location <i>Hill</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u><i>Annual Inspection at Crime Laboratory</i></u>	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): *Checked over*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Sgt. Ray Wedler</i>	Date <i>5-12-23</i>
---	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>[Signature]</i>	Certified Date <i>25 May 2023</i>
---	--------------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000           SN 80-006679  
Location = HILL                   8164.14.00 09/16  
05/12/2023                           20:29

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:           Enabled  
Start Test Sequence:       DABACABA  
Display Prelim Rslt?       Yes  
Display Third Digit?       Yes  
Inhib Printer(Y/N)?       No  
Display Volume?           No  
Disable On Memfull?       Yes  
# of Print Copies?         1  
Select Std (D/W/I)?       Dry  
Standard Value?           0.080  
Standard Lot #?           26021080A1  
Standard Cyl #?           18  
Standard Expiration?       10/05/2023  
Oper No?                   058405

Flow Cal. Date:           09/06/2016  
Slope                      701  
Intercept                  -682926

IR Calibration Date:       09/06/2016  
                                  3um                   9um

-----  
0th Coef(\*100):       -18983               -23165  
1st Coef(\*100):       280986               134481  
2nd Coef(\*100):       2443                   1290  
H2O adj(mg/l\*10k):   569                    553

\*\*\*\*\* Printer Test End \*\*\*\*\*

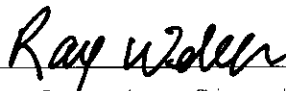

Operator Signature  
RAY WEBER

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006679  
Location = HILL      8164.14.00 09/16  
05/12/2023      21:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:07
02 Std. Gas	0.080	21:07
03 Room Air	0.000	21:07
04 Std. Gas	0.080	21:08
05 Room Air	0.000	21:08
06 Std. Gas	0.080	21:09
07 Room Air	0.000	21:09

Lot No = 26021080A1  
Cyl No = 18  
Exp Date = 10/05/2023  
County = 49      Oper No. = 058405

  
Operator Signature  
RAY WEBER

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006679  
Location = HILL      8164.14.00 09/16  
05/12/2023      21:22

Test	AC	Time
01 Room Air	0.000	21:25
02 *Subject Test	RFI*	21:26
03 Room Air	RFI*	21:26

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST TEST  
Sub DOB = 05/10/1985  
Sub Sex = Male      Weight = 225  
Test = DUI      Cit = 1234567  
Dr. Lic. = ND/TES051985  
Lot No = 26021080A1  
Cyl No = 18  
Expiration Date = 10/05/2023  
County = 49      Oper No. = 058405



Operator Signature  
RAY WEBER

Remarks:

Form 106-I8000