

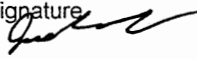


**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <b>80-002148</b>	Instrument Location <b>Ward City Jail</b>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Annual Inspection at Crime Laboratory</u>	

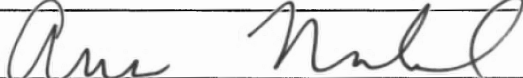
Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature  282	Date <b>05/17/2023</b>
----------------------------------------------------------------------------------------------------------------------	---------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date <b>18May2023</b>
--------------------------------------------------------------------------------------------------------------	------------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004188  
Location = WARD                8164.14.00 09/16  
05/17/2023                      15:08

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:                Enabled  
Start Test Sequence:            DABACABA  
Display Prelim Rslt?            Yes  
Display Third Digit?            Yes  
Inhib Printer(Y/N)?            No  
Display Volume?                No  
Disable On Memfull?            Yes  
# of Print Copies?              1  
Select Std (D/W/I)?            Dry  
Standard Value?                0.080  
Standard Lot #?                34921080A1  
Standard Cyl #?                37  
Standard Expiration?            02/05/2024  
Oper No?                        131334

Flow Cal. Date:                04/30/2020  
Slope                            645  
Intercept                       -637424

IR Calibration Date:            05/06/2020  
                                  3um                    9um

-----  
0th Coef(\*100):            -30092                -33449  
1st Coef(\*100):            293453                143538  
2nd Coef(\*100):            151                    602  
H2O adj(mg/l\*10k):        425                    462

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
JOSHUA GUDVANGEN

Remarks:

Form 106-I8000

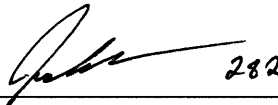
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = WARD      8164.14.00 09/16  
05/17/2023      15:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:09
02 Std. Gas	0.078	15:10
03 Room Air	0.000	15:10
04 Std. Gas	0.078	15:10
05 Room Air	0.000	15:11
06 Std. Gas	0.077	15:11
07 Room Air	0.000	15:12

Lot No = 34921080A1  
Cyl No = 37  
Exp Date = 02/05/2024  
County = 51      Oper No. = 131334

 282  
\_\_\_\_\_  
Operator Signature  
JOSHUA GUDVANGEN

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = WARD      8164.14.00 09/16  
05/17/2023      15:13

Test	AC	Time
01 Room Air	RFI*	15:14
02 Room Air	0.000	15:15

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST TEST  
Sub DOB = 01/01/2001  
Sub Sex = Unknown      Weight = NA  
Test = DUI      Cit = NA  
Dr. Lic. = ND/NA  
Lot No = 34921080A1  
Cyl No = 37  
Expiration Date = 02/05/2024  
County = 51      Oper No. = 131334

  
Operator Signature  
JOSHUA GUDVANGEN

Remarks:

Form 106-I8000