



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-006682	Instrument Location Divide County Sheriffs Department
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) Annual Inspection at Crime Laboratory	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): **NA**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 5/24/23
------------------------------------------------------------------------------------------------------------------	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date 24May2023
--------------------------------------------------------------------------------------------------------------	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = CROS 8164.14.00 09/16
05/24/2023 09:55

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 01923080A3
Standard Cyl #? 50
Standard Expiration? 02/05/2025
Oper No? 130831

Flow Cal. Date: 09/06/2016
Slope 701
Intercept -706193

IR Calibration Date:	09/06/2016	
	3um	9um

0th Coef(*100):	-17150	-23247
1st Coef(*100):	271726	134617
2nd Coef(*100):	831	1215
H2O adj(mg/l*10k):	393	518


***** Printer Test End *****



Operator Signature
MICHAEL DEHN

Remarks:

Form 106-I8000

Signature 
Date 5/24/2023
The undersigned, having custody of the original record, certifies that the information herein is a true and correct copy of the original document maintained as part of the activities of this agency.

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = CROS 8164.14.00 09/16
05/24/2023 09:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:58
02 Std. Gas	0.081	09:58
03 Room Air	0.000	09:59
04 Std. Gas	0.081	09:59
05 Room Air	0.000	10:00
06 Std. Gas	0.081	10:00
07 Room Air	0.000	10:01

Lot No = 01923080A3
Cyl No = 50
Exp Date = 02/05/2025
County = 12 Oper No. = 130831





Operator Signature
MICHAEL DEHN

Remarks:

Form 106-I8000

The undersigned, having custody of the original record, certifies that the information hereon is a true and correct copy of the original document maintained as part of the activities of this office.


Signature


Date


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = CROS 8164.14.00 09/16
05/24/2023 10:13

Test	AC	Time
01 Room Air	RFI*	10:16
02 Room Air	RFI*	10:16

*Invalid Test
Inhibited - RFI

Sub Name = RFI TEST, TEST TEST
Sub DOB = 01/01/1998
Sub Sex = Unknown Weight = NA
Test = OTH Cit = NA
Dr. Lic. = ND/NA
Lot No = 01923080A3
Cyl No = 50
Expiration Date = 02/05/2025
County = 12 Oper No. = 130831




Operator Signature
MICHAEL DEHN


Remarks:

Form 106-I8000

The undersigned, having custody of the original record, certifies that the information herein is a true and correct copy of the original document maintained as part of the activities of this office.

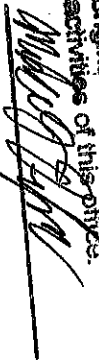

Signature



Date