

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Serial Number 80-007099 Instrument Location Tinga I	°D
Reason for Install/Repair	
☐ Install After Receiving From Crime Laboratory ☐ Install After Location Change	
Dother (Specify) Annual Inspection at Crime Libbratory	
Check When Done:	
1. Surge Protector Installed/Property Grounded.	
2. Telephone Line Connected to Intoxilyzer® 8000.	
3. Breath Tube Heated.	
4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).	
5. Scan/Enter Gas Cylinder Information (Level 1, Function S).	
Nexe X 6. Run Tests:	
6. Run Tests: A. Print Test (Level 1, Function P). B. ACA Test (Level 1, Function C).	
B. ACA Test (Level 1, Function C).	
C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio Du	uring Test).
7. Repair and/or Maintenance Performed (if any):	
8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the I	ntoxilyzer® for Use.
9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Age	ency.
 Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair C 104-G), Print Test, ACA Test, and RFI Test. 	Checkout (SFN59281, Form
Field Inspector Signature /	Date
ph re	6/15/23
Crime Laboratory Use Only	
This installation has been reviewed and the instrument is approved to be used for the analysis of breath to dete the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crit County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.	rmine alcohol concentration from me Laboratory Division, in the
Reviewed/Certified By Au Nole Note The Property of the Prop	Certified Date 15June2023

FORM 104-G

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007098
Location = TIOG 8164.16.00 09/18
06/15/2023 11:36

******* Printer Test *******

abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes Yes Display Third Digit? Inhib Printer(Y/N)? No NoDisplay Volume? Disable On Memfull?
of Print Copies? Yes 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 34921080A1 Standard Cyl #? 28 Standard Expiration? 02/05/2024 130977 Oper No?

Flow Cal. Date: 09/26/2018 Slope 684 Intercept -588446

IR Calibration Date: 09/26/2018

3um 9um

Oth Coef(*100): -40948 -15551

1st Coef(*100): 237555 134140

2nd Coef(*100): 2571 1433

H2O adj(mg/l*10k): 1097 457

****** Printer Test End *******

Operator Signature
JOSHUA NELSON

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 Location = TIOG

06/15/2023

SN 80-007098 8164.16.00 09/18 11:37

DRY CAL CHECK

Test		AC	Time
01 Room	Air	0.000	11:37
02 Std.	Gas	0.080	11:38
03 Room	Air	0.000	11:38
04 Std.	Gas	0.080	11:39
05 Room	Air	0.000	11:39
06 Std.	Gas	0.080	11:39
07 Room	Air	0.000	11:40

Lot No = 34921080A1

Cyl No = 28

Exp Date = 02/05/2024

County = 53

Oper No. = 130977

Operator Signature JOSHUA NELSON

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 Location = TIOG 06/15/2023

SN 80-007098 8164.16.00 09/18 11:42

Test			AC	Time
01	Room	Air	RFI*	11:44
02	Room	Air	RFI*	11:44

*Invalid Test Inhibited - RFI

Sub Name = TEST, TEST TEST Sub DOB = 01/01/2000

Sub Sex = Male

Weight = 125Test = DUI Cit = 000000001

Dr. Lic. = ND/TEST Lot No = 34921080A1

Cyl No = 28

Expiration Date = 02/05/2024

County = 53

Oper No. = 130977

Operator Signature JOSHUA NELSON

Remarks:

Form 106-I8000