



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Earl, William</i>		
Location <i>Lake Region Corr = DEVL</i>	Intoxilyzer® Serial Number <i>80-006671</i>	
Gas Lot Number <i>01923080A3</i>	Gas Cylinder Number <i>019</i>	Gas Expiration Date <i>02/05/2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0.080 AC
 B. 0.080 AC
 C. 0.080 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

All tests ran. AEN

Chemical Test Operator Signature <i>William Earl 4534</i>	Date <i>6/17/2023</i>
Reviewed By (Crime Laboratory Use Only) <i>Ane Nahl</i>	Date <i>27 June 2023</i>

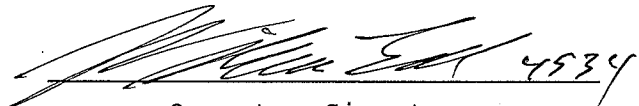
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006671
Location = DEVL 8164.14.00 09/16
06/17/2023 20:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:02
02 Std. Gas	0.080	20:02
03 Room Air	0.000	20:03
04 Std. Gas	0.080	20:03
05 Room Air	0.000	20:03
06 Std. Gas	0.080	20:04
07 Room Air	0.000	20:04

Lot No = 01923080A3
Cyl No = 19
Exp Date = 02/05/2025
County = 36 Oper No. = 132008


Operator Signature
WILLIAM EARL

Remarks:

Form 106-I8000