



**NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION**

**INTOXILYZER® 8000 REPAIR AND MAINTENANCE**

Intoxilyzer® 8000 Serial Number: 80-00 5360 Maintenance Location: TOXL

1. Reported Issue and Issue Found:

During annual inspection, the dry gas calibration values were high.

2. Repair/Maintenance Performed:

Calibration adjustment.

3. Does the instrument flow sensor need to be adjusted? Yes or No

- If Yes, complete Flow Sensor Adjustment and Calibration (Qualtrax ID: 11872), then go to step 4.
- If No, go to step 4.

4. Does the instrument calibration need to be adjusted? Yes or No

- If Yes, complete Intoxilyzer® 8000 Calibration Adjustment (Qualtrax ID: 11859), then go to step 5.
- If No, go to step 5.

5. Was the instrument cover removed for the repair? Yes or No

- If Yes, complete Intoxilyzer® 8000 Initial Inspection (Qualtrax ID: 11784), steps 6-7 are not needed, and sign below.
- If No, go to step 6.

6. Is an Annual Inspection due on the Intoxilyzer® 8000? Yes or No


- If Yes, complete Intoxilyzer® 8000 Annual Inspection (Qualtrax ID: 11698), step 7 is not needed, and sign below.
- If No, go to step 7.

7. Complete Intoxilyzer® 8000 Calibration (Qualtrax ID: 11871) and sign below.

Remarks/Notes: N/A

Instrument is acceptable to be used in the field.  Yes or No

If No, state reason(s) why:

  
Breath Analyst Signature

22 June 2023  
Date

  
Reviewer

22 June 2023  
Date