



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Pamela B. Pehl</i>		
Location <i>Stanton</i>	Intoxilyzer® Serial Number <i>80-004942</i>	
Gas Lot Number <i>14323080A1</i>	Gas Cylinder Number <i>021</i>	Gas Expiration Date <i>06/05/2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. 0. 092 AC  
 B. 0. 082 AC  
 C. 0. 082 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>P. Pehl</i>	Date <i>9/20/23</i>
Reviewed By (Crime Laboratory Use Only) <i>Ame Nahl</i>	Date <i>20Sep2023</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004942  
Location = STAT                    8164.14.00 09/16  
09/20/2023                            10:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:54
02 Std. Gas	0.082	10:54
03 Room Air	0.000	10:55
04 Std. Gas	0.082	10:55
05 Room Air	0.000	10:56
06 Std. Gas	0.082	10:56
07 Room Air	0.000	10:57

Lot No = 14323080A1  
Cyl No = 21  
Exp Date = 06/05/2025  
County = 29                            Oper No. = 120906



Operator Signature  
PAMELA PEHL

Remarks:

Form 106-I8000