



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Alex Cook</i>		
Location <i>Renville County - Mohall</i>	Intoxilyzer® Serial Number <i>80-006505</i>	
Gas Lot Number <i>14323080A1</i>	Gas Cylinder Number <i>23</i>	Gas Expiration Date <i>6/5/2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0.081 AC
 B. 0.078 AC
 C. 0.078 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>Alex Cook</i>	Date <i>10/3/2023</i>
Reviewed By (Crime Laboratory Use Only) <i>Ame Nahl</i>	Date <i>10/4/2023</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006505
Location = MOHL 8164.14.00 09/16
10/03/2023 14:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:44
02 Std. Gas	0.081	14:44
03 Room Air	0.000	14:45
04 Std. Gas	0.078	14:45
05 Room Air	0.000	14:46
06 Std. Gas	0.078	14:46
07 Room Air	0.000	14:47

Lot No = 14323080A1
Cyl No = 23
Exp Date = 06/05/2025
County = 38 Oper No. = 133141



Operator Signature
ALEX COOK

Remarks:

Form 106-I8000