



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Troy White Owl</i>		
Location <i>105 500 Place</i>	Intoxilyzer® Serial Number <i>80-004498</i>	
Gas Lot Number <i>1432 3080A1</i>	Gas Cylinder Number <i>72</i>	Gas Expiration Date <i>6-05-25</i>

Check When Done:

*aw*

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S)
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. *0.081* AC  
 B. *0.081* AC  
 C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:  
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G)  
 B. ACA Test Record.

*aw*

Chemical Test Operator Signature <i>Troy White Owl</i>	Date <i>10-10-2023</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>12 Oct 2023</i>

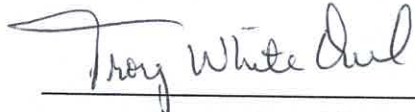
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006498  
Location = NTPD      8164.14.00 09/16  
10/10/2023      08:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:39
02 Std. Gas	0.081	08:39
03 Room Air	0.000	08:39
04 Std. Gas	0.081	08:40
05 Room Air	0.000	08:40
06 Std. Gas	0.081	08:41
07 Room Air	0.000	08:41

Lot No = 14323080A1  
Cyl No = 72  
Exp Date = 06/05/2025  
County = 31      Oper No. = 020626

  
\_\_\_\_\_  
Operator Signature  
TROY WHITEOWL

Remarks:

Form 106-I8000