



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>MICHAEL MAROLT</i>		
Location <i>NAPO</i>	Intoxilyzer® Serial Number <i>80-004200</i>	
Gas Lot Number <i>1432308044</i>	Gas Cylinder Number <i>017</i>	Gas Expiration Date <i>6-5-2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.081* AC
 B. *0.082* AC
 C. *0.082* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Michael Marolt</i>	Date <i>12-19-23</i>
Reviewed By (Crime Laboratory Use Only) <i>Aime Nahl</i>	Date 26Dec2023


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = NAPO 8164.14.00 09/16
12/19/2023 15:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:01
02 Std. Gas	0.081	15:02
03 Room Air	0.000	15:02
04 Std. Gas	0.082	15:02
05 Room Air	0.000	15:03
06 Std. Gas	0.082	15:03
07 Room Air	0.000	15:04

Lot No = 14323080A4
Cyl No = 17
Exp Date = 06/05/2025
County = 24 Oper No. = 131100


Operator Signature
MICHAEL MAROLT

Remarks:

Form 106-I8000