

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Serial Number 80 - 006504	Instrument Location CARR	
Reason for Install/Repair		
X Install After Receiving From Crime Laboratory Install	After Location Change	
Other (Specify)		
Check When Done:		
I. Surge Protector Installed/Property Grounded.		
2. Telephone Line Connected to Intoxilyzer® 8000.		
X 3. Breath Tube Heated.	./0	
4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and L	ocation; Level 2, Function E). N/17	
5. Scan/Enter Gas Cylinder Information (Level 1, Function	s).	
⊠ 6. Run Tests:		
A. Print Test (Level1, Function P).		
B. ACA Test (Level 1, Function C).		
C. Radio Frequency Interference (RFI) Test (CMS Mo	de or Level 1, Function B or C; Key Radio Du	ring Test).
7. Repair and/or Maintenance Performed (if any):	N/A	
8. Complete the Tap Partion of the Intoxilyzer® Record (S		
9. File Previous Intoxilyzer® Record (SFN504096, Form	120-G) at the Intoxilyzer® Location at the Age	incy.
 9. File Previous intoxity2ers Record (Si Record C, Forman, 10. Send the Following to the Crime Laboratory: Complete 104-G), Print Test, ACA Test, and RFI Test. 	ed Intoxilyzer® 8000 Installation and Repair C	heckout (SFN59281, Form
Field Inspector Signature	1621	Date 12/15/2023
Crime Laboratory Use Only	the second s	mine alcohol concentration from
This installation has been reviewed and the instrument is approve the date the Field Inspector performed the installation. This recor County of Burleigh, North Dakota, is certified to be a true and corr	a on me at the Onice of Attorney General, on	me Laboratory Division, in the
Reviewed/Certified By	1	Certified Date
In In In	1	OzJanzozy

Talked w/ Chrs. He couldn't initially get the contacting me, I walked him through it	password to work. after Received a confirmation
email on D Jan 2, 2024 that the password	worked.
email on D Jan 212024 - Mar me powered	Worked anna Narehood

FORM 104-G

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000
 SN 80-006504

 Location = CARR
 8164.14.00 09/16
 12/15/2023 13:46 abcdefghijklmnopgrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Yes Display Third Digit? Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? Standard Cyl #? 01923080A3 25 Standard Expiration? 02/05/2025 Oper No? 131530 Flow Cal. Date: 08/20/2015 Slope 681 Intercept -673151 IR Calibration Date: 05/31/2017 3um 9um ------0th Coef(*100): -21488 1st Coef(*100): 265581 2nd Coef(*100): 2507 -16160 137324 1149 H2O adj(mg/l*10k): 519 348 ********** Printer Test End **********

1621

Operator Signature CHRISTOPHER ORMISTON

Remarks:

Print Test

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006504Location = CARR8164.14.00 09/1612/15/202313:50

	DRY CAL CHECK	
Test	AC	Time
01 Room Ai 02 Std. Ga 03 Room Ai 04 Std. Ga 05 Room Ai 06 Std. Ga 07 Room Ai	s 0.080 r 0.000 s 0.081 r 0.000 s 0.080	13:51 13:51 13:52 13:52 13:52 13:53 13:53

Lot No = 01923080A3 Cyl No = 25 Exp Date = 02/05/2025 County = 16 Ope

Oper No. = 131530

1621

Operator Signature CHRISTOPHER ORMISTON

Remarks:

ACA Test

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006504Location = CARR8164.14.00 09/1612/15/202313:48

	DRY CAL CHECK	
Test	AC	Time
01 Room Air	RFI*	13:49
02 Room Air	0.000	13:49

*Invalid Test Inhibited - RFI

1621

Operator Signature CHRISTOPHER ORMISTON

Remarks:

Test

Form 106-18000

ł