



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <i>80-003067</i>	Instrument Location <i>Walsh County LEC</i>
Reason for Install/Repair	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Jacob Ribba</i>	Date <i>1/3/24</i>
---	-----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Aime Nahl</i>	Certified Date <i>16Jan2024</i>
---	------------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003067  
Location = GFTN      8164.14.00 09/16  
01/03/2024      18:10

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&\*()\_+?

Current Instrument Setup  
Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On Memfull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 34921080A1  
Standard Cyl #? 36  
Standard Expiration? 02/05/2024  
Oper No? 132597

Flow Cal. Date: 04/17/2020  
Slope 609  
Intercept -236750

IR Calibration Date: 04/17/2020  
3um      9um

-----  
0th Coef(\*100):      -19307      -18350  
1st Coef(\*100):      282277      142540  
2nd Coef(\*100):      1257      744  
H2O adj(mg/l\*10k): 302      214

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
JACOB NIKKO

Remarks:

The undersigned, having custody of the original public record, certifies that the information hereon is a true and correct copy of the original document maintained as a part of the activities of this agency.

Signature/Title Jacob Nikko / Police Officer  
Date Grafton PD / 1/3/24

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

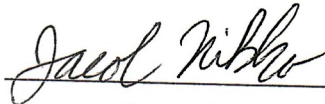
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003067  
Location = GFTN      8164.14.00 09/16  
01/03/2024      18:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:05
02 Std. Gas	0.079	18:05
03 Room Air	0.000	18:06
04 Std. Gas	0.079	18:06
05 Room Air	0.000	18:07
06 Std. Gas	0.079	18:07
07 Room Air	0.000	18:08

Lot No = 34921080A1  
Cyl No = 36  
Exp Date = 02/05/2024  
County = 50

Oper No. = 132597

  
\_\_\_\_\_  
Operator Signature  
JACOB NIKKO

Remarks:

Form 106-I8000

I, the undersigned, having a copy of the original public record, certifies that the information hereon is a true and correct copy of the original document maintained as a part of the activities of this agency.

Signature/Title Jacob Nikko / Police Officer  
Agency/Date Grafton PD / 1/3/24

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer  
North Dakota Model 8000  
Location = GFTN  
01/03/2024

Alcohol Analyzer  
SN 80-003067  
8164.14.00 09/16  
18:13

DRY CAL CHECK

Test	AC	Time
01 Room Air		18:13
02 Room Air	RFI* 0.000	18:14
*Invalid Test Inhibited - RFI		

Lot No = 34921080A1  
Cyl No = 36  
Exp Date = 02/05/2024  
County = 50

Oper No. = 132597

*Jacob Nikko*

Operator Signature  
JACOB NIKKO

Remarks:

Form 106-I8000

The Undersigned, having custody of the original public record, certifies that the information hereon is a true and correct copy of the original document maintained as a part of the activities of this agency.

Signature/Title *Jacob Nikko / Police Officer*  
Agency/Date *Grafton PD / 1/3/24*