



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Timothy Bleth</i>		
Location <i>BSPD</i>	Intoxilyzer® Serial Number <i>80-005945</i>	
Gas Lot Number <i>14323080 A4</i>	Gas Cylinder Number <i>8</i>	Gas Expiration Date <i>06/05/2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.081* AC
 - B. *0.081* AC
 - C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>T. Bleth</i>	Date <i>1/10/2024</i>
Reviewed By (Crime Laboratory Use Only) <i>Aime Nahl</i>	Date 10Jan2024

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = BSPD 8164.14.00 09/16
01/10/2024 10:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:44
02 Std. Gas	0.081	10:44
03 Room Air	0.000	10:45
04 Std. Gas	0.081	10:45
05 Room Air	0.000	10:45
06 Std. Gas	0.081	10:46
07 Room Air	0.000	10:46

Lot No = 14323080A4
Cyl No = 8
Exp Date = 06/05/2025
County = 08 Oper No. = 069823



Operator Signature
TIMOTHY BLETH

Remarks: *New cylinder change ACA - OK*

Form 106-I8000