



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Trevor K. Bergerson</i>		
Location <i>BOPD</i>	Intoxilyzer® Serial Number <i>80-007093</i>	
Gas Lot Number <i>14323080A4</i>	Gas Cylinder Number <i>002</i>	Gas Expiration Date <i>06/05/2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. 0. *082* AC  
 B. 0. *081* AC  
 C. 0. *081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>T. Bergerson #1821</i>	Date <i>01/13/2024</i>
Reviewed By (Crime Laboratory Use Only) <i>Aime Nahl</i>	Date 16Jan2024


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007093  
Location = BOPD                    8164.16.00 09/18  
01/13/2024                            16:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:07
02 Std. Gas	0.082	16:08
03 Room Air	0.000	16:08
04 Std. Gas	0.081	16:09
05 Room Air	0.000	16:09
06 Std. Gas	0.081	16:10
07 Room Air	0.000	16:10

Lot No = 14323080A4  
Cyl No = 2  
Exp Date = 06/05/2025  
County = 06                            Oper No. = 130538

 #1821  
Operator Signature  
TREVOR BERGERSON

Remarks:  
*Gas Cylinder Install*

Form 106-I8000