



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Nick Schoedel</i>		
Location <i>BMDc</i> Burleigh County Sheriff's Department	Intoxilyzer® Serial Number <i>80-009950</i>	
Gas Lot Number <i>01923080A3</i>	Gas Cylinder Number <i>53</i>	Gas Expiration Date <i>02-05-2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.084* AC
 - B. *0.083* AC
 - C. *0.082* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Nick Schoedel</i>	Date <i>1/17/24</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>1/17/24</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = BMDC 8164.14.00 09/16
01/17/2024 10:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:05
02 Std. Gas	0.084	10:06
03 Room Air	0.000	10:06
04 Std. Gas	0.083	10:06
05 Room Air	0.000	10:07
06 Std. Gas	0.082	10:07
07 Room Air	0.000	10:08

Lot No = 01923080A3
Cyl No = 53
Exp Date = 02/05/2025
County = 08 Oper No. = 131854


Operator Signature
NICK SCHOEDEL

Remarks:

Form 106-I8000