



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Earl, William</i>		
Location <i>DEVL</i>	Intoxilyzer® Serial Number <i>80-006671</i>	
Gas Lot Number <i>14323080A4</i>	Gas Cylinder Number <i>1</i>	Gas Expiration Date <i>6/9/2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits. Ex. 0.081).
 - A. 0.083 AC
 - B. 0.083 AC
 - C. 0083 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>1/23/2024</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i> 4534	Date 25Jan2024


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006671
Location = DEVL 8164.14.00 09/16
01/22/2024 00:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	00:53
02 Std. Gas	0.083	00:53
03 Room Air	0.000	00:54
04 Std. Gas	0.082	00:54
05 Room Air	0.000	00:54
06 Std. Gas	0.083	00:55
07 Room Air	0.000	00:55

Lot No = 14323080A4
Cyl No = 1
Exp Date = 06/05/2025
County = 36 Oper No. = 132008

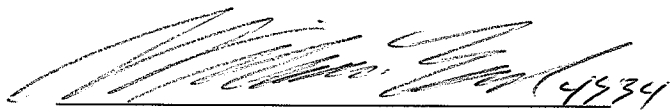

Operator Signature
WILLIAM EARL

Remarks:

Form 106-I8000