

## ETHANOL GAS STANDARD CYLINDER REPORT NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59282 (08/2018)

Chemical Test Operator Name (Print)	RD		
POWERS LAKE PL	)	Intoxilyzer® Serial No 80 - 00 66	
Gas Lot Number 1432308044	Gas Cylinder Number	7	Gas Expiration Date 06-05-2025

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

- A. 0.083 AC
- B. 0.083 AC
- C. 0.08 3 AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

5. Send the Following to the Crime Laboratory:

- A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
- B. ACA Test Record.

Chemical Test Operator Signature	Date 01-16-2024
Reviewed By (Crime Laboratory Use Only)	Date 12Feb2024

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer<br/>North Dakota Model 8000Alcohol Analyzer<br/>SN 80-006687Location = PLPD8164.14.00 09/1601/16/202415:40

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.083 0.000 0.083 0.000 0.083 0.000	15:41 15:42 15:42 15:43 15:43 15:44 15:44

Operator Signature BRYTON ALLARD

Remarks:

Form 106-18000

.