



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>GONZALEZ, JORGE F.</i>		
Location <i>37 Lisbon PD</i>	Intoxilyzer® Serial Number <i>80-007086</i>	
Gas Lot Number <i>14323080A4</i>	Gas Cylinder Number <i>9</i>	Gas Expiration Date <i>06-05-2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. *082* AC
 - B. 0. *082* AC
 - C. 0. *082* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Jorge Gonzalez</i>	Date <i>02-02-24</i>
Reviewed By (Crime Laboratory Use Only) <i>Ame Nahl</i>	Date <i>05Feb2024</i>

Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000 SN 80-007086
 Location = LISB 8164.16.00 09/18
 02/02/2024 16:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:53
02 Std. Gas	0.082	16:53
03 Room Air	0.000	16:54
04 Std. Gas	0.082	16:54
05 Room Air	0.000	16:55
06 Std. Gas	0.082	16:55
07 Room Air	0.000	16:55

Lot No = 14323080A4
 Cyl No = 9
 Exp Date = 06/05/2025
 County = 37 Oper No. = 031021

Jorge Gonzalez

 Operator Signature
 JORGE GONZALEZ

Remarks:

ACA TEST.

Form 106-I8000