

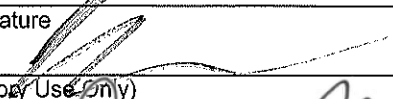
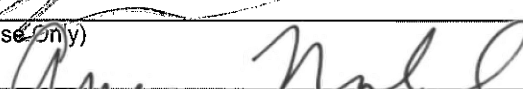


ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) GALLAGHER, JONATHAN D.		
Location GARR	Intoxilyzer® Serial Number 80-002669	
Gas Lot Number 14323080A1	Gas Cylinder Number 74	Gas Expiration Date 06-05-2025

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.080 AC
 - B. 0.080 AC
 - C. 0.080 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date 3/18/2024
Reviewed By (Crime Laboratory Use Only) 	Date 19Mar2024

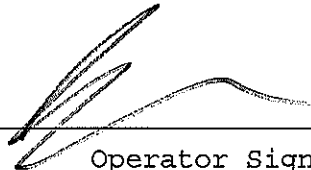
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = GARR 8164.14.00 09/16
03/18/2024 21:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	22:00
02 Std. Gas	0.080	22:00
03 Room Air	0.000	22:01
04 Std. Gas	0.080	22:01
05 Room Air	0.000	22:02
06 Std. Gas	0.080	22:02
07 Room Air	0.000	22:03

Lot No = 14323080A1
Cyl No = 74
Exp Date = 06/05/2025
County = 28 Oper No. = 131749


Operator Signature
JONATHAN GALLAGHER

Remarks:

Form 106-I8000