

## INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Serial Number 00-0057002	Instrument Location MCF			
Reason for Install/Repair				
☐ Install After Receiving From Crime Laboratory ☐ Install After Location Change				
& Other (Specify) annual Inspection at chima Laboratory				
	- U			
Check When Done:				
1. Surge Protector Installed/Property Grounded.				
2. Telephone Line Connected to Intoxilyzer® 8000.				
3. Breath Tube Heated.				
4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).				
5. Scan/Enter Gas Cylinder Information (Level 1, Function S).				
6. Run Tests:				
A. Print Test (Level1, Function P).				
B. ACA Test (Level 1, Function C).				
C. Radio Frequency Interference (RFI) Test (CMS Mode or	Level 1, Function B or C; Key Radio Du	ring Test).		
7. Repair and/or Maintenance Performed (if any):				
8. Complete the Top Portion of the Intoxilyzer® Record (SFN5)	0496, Form 120-G) and Place it by the Ir	toxilyzer® for Use.		
☑ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.				
10. Send the Following to the Crime Laboratory: Completed Int 104-G), Print Test, ACA Test, and RFI Test.	oxilyzer® 8000 Installation and Repair C	neckout (SFN59281, Form		
Field Inspector Signature		Date 04 02 24		
Crime Laboratory Use Only				
This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.				
Reviewed/Certified By	0	Certified Date 05April2024		

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = MCSO 8164.14.00 09/16
04/02/2024 13:22

abcdefghijklmnopqrstuvwxyz1234567890-= |
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +?

Current Instrument Setup

Data Entry Mode: Enabled DABACABA Start Test Sequence: Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer (Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 14323080A4 Standard Cyl #? 15 Standard Expiration? 06/05/2025 Oper No? 132007

Flow Cal. Date: 05/22/2020 Slope 659 Intercept -620990

IR Calibration Date: 05/26/2020 3um 9um

Oth Coef(\*100): -20883 -25370

1st Coef(\*100): 265508 144960

2nd Coef(\*100): 936 714

H2O adj(mq/l\*10k): 391 379

\*\*\*\*\*\* Printer Test End \*\*\*\*\*\*\*

Operator SignatAFE OF NORTH DAKOTA KARI STUAREOUNTY OF MCKENZIE

Remarks:

I certify that the foregoing is a true and correct copy of the original document on file in this office.

Dated at Watford City, North Dakota.

on this ( ) day of.

oodling CALLA

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = MCSO
04/03/2024

Alcohol Analyzer SN 80-005362 8164.14.00 09/16 06:46

## DRY CAL CHECK

Te	est		AC	Time
01	Room	Air	0.000	06:47
02	Std.	Gas	0.082	06:47
03	Room	Air	0.000	06:48
04	Std.	Gas	0.082	06:48
05	Room	Air	0.000	06:49
06	Std.	Gas	0.081	06:49
07	Room	Air	0.000	06:50

Lot No = 14323080A4

Cyl No = 15

Exp Date = 06/05/2025

County = 27

Oper No. = 132007

Operator Signature KARI STUART

Remarks:

Form 106-I8000

STATE OF NORTH DAKOTA COUNTY OF MCKENZIE

I certify that the foregoing is a true and correct copy of the original document on file in this office. Dated at Watford City, North Dakota.

nithis /\//

\_day of

20. W

de County Sheriff's Department

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = MCSO 04/03/2024

Alcohol Analyzer SN 80-005362 8164.14.00 09/16 06:50

Test			AC	Time
01	Room	Air	RFI*	06:51
02	Room	Air	RFI*	06:51

\*Invalid Test Inhibited - RFI

Sub Name = STUART, KARI A

Sub DOB = 09/09/1990

Sub Sex = FemaleWeight = 157Test = OTH Cit = 0000000000

Dr. Lic. = ND/STU900000

Lot No = 14323080A4

Cyl No = 15

Expiration Date = 06/05/2025

County = 27

Oper No. = 132007

Operator Signature KARI STUART

Remarks:

Form 106-I8000

STATE OF NORTH DAKOTA COUNTY OF MCKENZIE

I certify that the foregoing is a true and correct copy of the original document on file in this office.

Dated at Watford City, North Dakota.

's Department