



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <b>80-005945</b>	Instrument Location <b>BSPD</b>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Annual Inspection at Crime Laboratory</u>	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>T. Blum</i>	Date <b>4-5-24</b>
---	-----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Aime Wahl</i>	Certified Date <b>05April2024</b>
---	--------------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-005945  
Location = BSPD               8164.14.00 09/16  
04/05/2024                     14:17

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

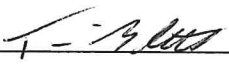
Data Entry Mode:           Enabled  
Start Test Sequence:       DABACABA  
Display Prelim Rslt?       Yes  
Display Third Digit?       Yes  
Inhib Printer(Y/N)?       No  
Display Volume?           No  
Disable On Memfull?       Yes  
# of Print Copies?         1  
Select Std (D/W/I)?       Dry  
Standard Value?           0.080  
Standard Lot #?           14323080A4  
Standard Cyl #?           8  
Standard Expiration?      06/05/2025  
Oper No?                   069823

Flow Cal. Date:           06/20/2013  
Slope                      637  
Intercept                 -566407

IR Calibration Date:      05/22/2015  
                            3um                   9um

-----  
0th Coef(\*100):       -21213               -23093  
1st Coef(\*100):       279171               139292  
2nd Coef(\*100):       2898                   1291  
H2O adj(mg/l\*10k):   332                   319

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
-----  
Operator Signature  
TIMOTHY BLETH

Remarks: *Print Test - New Install from Annual Inspection*


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005945  
Location = BSPD                    8164.14.00 09/16  
04/05/2024                            14:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:26
02 Std. Gas	0.081	14:26
03 Room Air	0.000	14:27
04 Std. Gas	0.081	14:27
05 Room Air	0.000	14:28
06 Std. Gas	0.081	14:28
07 Room Air	0.000	14:28

Lot No = 14323080A4  
Cyl No = 8  
Exp Date = 06/05/2025  
County = 08                            Oper No. = 069823

  
\_\_\_\_\_  
Operator Signature  
TIMOTHY BLETH

Remarks:

*ACA - New Install from Annual Inspection*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

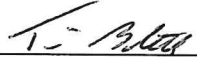
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005945  
Location = BSPD                      8164.14.00 09/16  
04/05/2024                              14:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:31
02 Std. Gas	RFI*	14:31
03 Room Air	0.000	14:32

\*Invalid Test  
Inhibited - RFI

Lot No = 14323080A4  
Cyl No = 8  
Exp Date = 06/05/2025  
County = 08                              Oper No. = 069823

  
\_\_\_\_\_  
Operator Signature  
TIMOTHY BLETH

Remarks:  
*RFI test - new install from Annual Inspection*  
*OK - Turned radio on & RFI detected. Did not have to key the mic.*  
Form 106-I8000