



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <b>Terry Dillon</b>		
Location <b>CARS</b>	Intoxilyzer® Serial Number <b>80-006681</b>	
Gas Lot Number <b>14323080A1</b>	Gas Cylinder Number <b>#65</b>	Gas Expiration Date <b>06/05/2025</b>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. 0.**079** AC  
 B. 0.**079** AC  
 C. 0.**079** AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Terry Dillon</i>	Date <b>04/08/2024</b>
Reviewed By (Crime Laboratory Use Only) <i>Aime Nahl</i>	Date <b>09April2024</b>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006681  
Location = CARS      8164.14.00 09/16  
04/08/2024      12:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:36
02 Std. Gas	0.079	12:36
03 Room Air	0.000	12:37
04 Std. Gas	0.079	12:37
05 Room Air	0.000	12:38
06 Std. Gas	0.079	12:38
07 Room Air	0.000	12:39

Lot No = 14323080A1  
Cyl No = 65  
Exp Date = 06/05/2025  
County = 19      Oper No. = 132318

  
\_\_\_\_\_  
Operator Signature  
TERRY DILLON

Remarks:

Form 106-I8000