



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Landon Jacobson</i>		
Location <i>Williston (Jail) [WILL]</i>	Intoxilyzer® Serial Number <i>80-003060</i>	
Gas Lot Number <i>28423080A3</i>	Gas Cylinder Number <i>38</i>	Gas Expiration Date <i>11-05-2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0.080 AC
 B. 0.081 AC
 C. 0.081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Landon Jacobson</i>	Date <i>04-14-2024</i>
Reviewed By (Crime Laboratory Use Only) <i>Aime Null</i>	Date <i>17 April 2024</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003060
Location = WILL 8164.14.00 09/16
04/14/2024 21:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:55
02 Std. Gas	0.080	21:56
03 Room Air	0.000	21:56
04 Std. Gas	0.081	21:57
05 Room Air	0.000	21:57
06 Std. Gas	0.081	21:58
07 Room Air	0.000	21:58

Lot No = 28423080A3
Cyl No = 38
Exp Date = 11/05/2025
County = 53 Oper No. = 133689


Operator Signature
LANDON JACOBSON

Remarks:

Form 106-I8000