



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

| | | |
|---|--|--|
| Chemical Test Operator Name (Print) <i>Earl, William J</i> | | |
| Location <i>FTOT</i> | Intoxilyzer® Serial Number <i>80-005358</i> | |
| Gas Lot Number <i>28423080A3</i> | Gas Cylinder Number <i>48</i> | Gas Expiration Date <i>11/05/2025</i> |

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.080* AC
 B. *0.079* AC
 C. *0.080* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

| | |
|--|--------------------------|
| Chemical Test Operator Signature <i>William J Earl</i> | Date <i>4/16/2024</i> |
| Reviewed By (Crime Laboratory Use Only) <i>Aime Nabel</i> | Date 18 April 2024 |


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005358
Location = FTOT 8164.14.00 09/16
04/16/2024 14:36

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:38 |
| 02 Std. Gas | 0.080 | 14:38 |
| 03 Room Air | 0.000 | 14:38 |
| 04 Std. Gas | 0.079 | 14:39 |
| 05 Room Air | 0.000 | 14:39 |
| 06 Std. Gas | 0.080 | 14:40 |
| 07 Room Air | 0.000 | 14:40 |

Lot No = 28423080A3
Cyl No = 48
Exp Date = 11/05/2025
County = 62 Oper No. = 132008

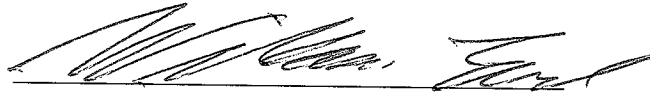

Operator Signature
WILLIAM EARL

Remarks:

Form 106-I8000