



NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00 5939

A. General Setup and Checks:

1. Initial start up
  - Power On
  - 5-minute countdown during initial warm-up
  - 1-minute countdown to begin test after standby mode
2.  Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
  - Level 1 (Operator)
  - Level 2 (Field Inspector)
  - Level 3 (Lab Personnel)
4.  Printer Setup (Level 3,S,P)
5.  Forms Load Completed (CMI, Inc., CoBrA Software)

B. Level 1 Setup and Checks:

1.  Internal Printer Test (Level 1,P). Sign and attach record.
  - Paper Feed
2.  External Printer Test (Level 1,P). Sign and attach record.
3. Barcode Reader (Level 1,B) (Tone sounds after each)
  - Operator ID
  - Invalid ID
  - ND License
  - R key: Refusal. Sign and attach record.

C. Level 2 Setup and Checks:

1.  Continuous Air Blank (Level 2,A)
2.  Diagnostic (Level 2,D). Sign and attach test record.
3.  Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
  - Inst Ser#: 80-005939 DSP Ver: 1362.14
  - Core Ver: 81164.14 CPLD Ver: 7

D. Level 3 Setup and Checks:

1. DVM Monitor (Level 3,D,D):

3: 12724 s: 1 C: 47.00  
9: 12809 s: 1 B: 45

DVM (3 & 9) between 9,000-16,000 counts

RFI indicator with keyed radio: 091

2. Function (F) Keys:

F5 – Tone

F9 - Fan

F6 – Green LED

F10 – Pump

F7 – Red LED

F11 – Solenoid

3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 950 CF: 1.04

4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 246 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T):

C: 47.0 A: 47.5 I: 0.0

B: 45.0 V: 0.0

Cell Temperature (C) between  $47 \pm 0.02$  °C

Breath Hose Temperature (B) between 35.5 – 47 °C

Internal Ambient Temperature (A) above 41 °C

6. Tank Monitor (Level 3,D,G):

Display: 28 psi Regulator: 10 psi

Display and Regulator  $\pm 50$  psi of each other Yes or No

Gas tank tare necessary? Yes or No

If Yes, display readings after tare (Level 3,M,C,G):

Display: \_\_\_\_\_ psi Regulator: \_\_\_\_\_ psi

7.  General Setup (Level 3,S,G)

8.  Configure Start Test (Level 3,S,T)

9.  Network (Level 3,S,N)

E. Tests (Sign and attach all test records):

1.  Configure wet bath simulator for the following test (Level 1,S)

Zero AC-Wet Calibration Check (Level 1,C):

Sim ser #: MP5321 Lot #: NA Exp date: N/A

Known value: 0.000 AC

Results are Zero AC

2.  Configure wet bath simulator for the following test (Level 1,S)

Low AC-Wet Calibration Check (Level 1,C):

Known Value  $\leq 0.03$  AC: 0.020 AC

Sim ser #: MP3002 Lot #: 202301E Exp date: 17Jan25

Results  $\pm 0.005$  of known AC

3.  Configure wet bath simulator for the following test (Level 1,S)  
 High AC-Wet Calibration Check (Level 1,C):  
 Known Value  $\geq 0.25$  AC: 0.040 AC 0.400 acn  
 Sim ser #: MP6036 Lot #: 202212 Exp date: 08Dec24  
 Results  $\pm 5\%$  of known AC
4.  Configure dry gas standard for the following tests (Level 1,S)  
 Known Value: 0.080 AC  
 Gas Cylinder Lot #: 01923080A3 Cylinder #: 17 Exp. Date: 215/25
5. 0.10% Ethanol + 0.05% Acetone (Level 1,B):  
 Sim. Ser #: DR7344 Lot #: 1658 Exp. Date: NIA  
 Display reads "Interferent Detect"
6. 0.05% Acetone (Level 1,B):  
 Sim. Ser #: DR7344 Lot #: ACE3 Exp. Date: NIA  
 Display reads "Interferent Detect"
7. 0.10% Methanol (Level 1,B):  
 Sim. Ser #: DR7345 Lot #: MET3 Exp. Date: NIA  
 Display Reads "Interferent Detect"
8. 0.10% Isopropanol (Level 1,B):  
 Sim. Ser #: DR7350 Lot #: 1503 Exp. Date: NIA  
 Display Reads "Interferent Detect"
9.  Ambient Fail (Level 1,B): Display read and test record correct.
10.  Invalid Sample (Level 1,B): Display read and test record correct.
11.  Sequence Aborted (Level 1,B): Display read test record correct.
12.  ABA (Level 1,B): Test record correct.
13.  Calibration check out of tolerance (CMS mode): Display read and test record correct.
14.  RFI Detected (B,C, or CMS mode): Display read and test record correct.
15.  Tank Pressure Below Minimum (CMS mode): Display reads correct.
16.  Deficient Samples (CMS mode): Display read and test record correct.
17.  Improper Sample (CMS mode): Display read and test record correct.
18.  Difference Too Great (CMS mode): Display read and test record correct.
19.  CMS (CMS mode): Test record correct.
20. Stability Test (Level 3,T,S):  
 Ave: 0.0805 Std Dev: 0.0006 Rel Std Dev: 0.76%

Maintenance/Repairs/Notes: NIA

---



---




---

Instrument is acceptable to be used in the field.  
If No, state reason(s) why:

Yes or No

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

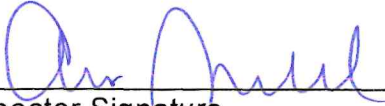  
\_\_\_\_\_  
Inspector Signature

17 Apr 2024  
\_\_\_\_\_  
Date

  
\_\_\_\_\_  
Reviewer

18 Apr 2024  
\_\_\_\_\_  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-005939  
Location = TOXL 8164.14.00 09/16  
04/17/2024 08:02

\*\*\*\*\* Printer Test \*\*\*\*\*

12 char/line  
abcdefghijkl  
ABCDEFGHIJKL

16 char per line  
abcdefghijklmnop  
ABCDEFGHIJKLMN

21 chars per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMNQRSTU

24 characters per line  
abcdefghijklmnopqrstuwx  
ABCDEFGHIJKLMNQRSTUWX

32 characters per line  
abcdefghijklmnopqrstuwx123456  
ABCDEFGHIJKLMNQRSTUWXYZ123456

42 characters per line  
abcdefghijklmnopqrstuwx1234567890abcdef  
ABCDEFGHIJKLMNQRSTUWXYZ1234567890ABCDEF

Current Instrument Setup

Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On MemFull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 34921080A1  
Standard Cyl #? 33  
Standard Expiration? 02/05/2024  
Oper No? 133237

Flow Cal. Date: 06/18/2020  
Slope 611  
Intercept -592495

IR Calibration Date: 06/18/2020  
3um 9um

0th Coef(*100):	-15726	-19510
1st Coef(*100):	296627	143165
2nd Coef(*100):	-369	494
H2O adj(mg/l*10K):	220	240

\*\*\*\*\* Test End \*\*\*\*\*



Operator Signature  
ANNA NAREHOOD

Remarks:

Internal Print  
Test  
Form 106-18000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-005939  
Location = TOXL                              8164.14.00 09/16  
04/17/2024                                      08:05

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup  
Data Entry Mode:                      Enabled  
Start Test Sequence:                      DABACABA  
Display Prelim Rslt?                      Yes  
Display Third Digit?                      Yes  
Inhib Printer(Y/N)?                      No  
Display Volume?                          No  
Disable On Memfull?                      Yes  
# of Print Copies?                          1  
Select Std (D/W/I)?                      Dry  
Standard Value?                          0.080  
Standard Lot #?                              34921080A1  
Standard Cyl #?                              33  
Standard Expiration?                      02/05/2024  
Oper No?                                      133237

Flow Cal. Date:                              06/18/2020  
Slope    611  
Intercept                                      -592495

IR Calibration Date:                      06/18/2020  
    3um                                      9um

-----  
0th Coef(\*100):                      -15726                                      -19510  
1st Coef(\*100):                      296627                                      143165  
2nd Coef(\*100):                      -369    494  
H2O adj(mg/l\*10k): 220                                      240

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
ANNA NAREHOOD

Remarks: External Print Test


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005939  
Location = TOXL                    8164.14.00 09/16  
04/17/2024                            08:05

Test	AC	Time
01 Room Air	0.000	08:06
02 *Subject Test	REF*	08:06
03 Room Air	0.000	08:06

\*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = OTH                            Cit = REFUSAL  
Dr. Lic. = ND/DIS821456  
Lot No = 34921080A1  
Cyl No = 33  
Expiration Date = 02/05/2024  
County = 08                            Oper No. = 133237

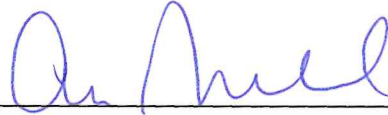

Operator Signature  
ANNA NAREHOOD

Remarks: Refusal

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      08:10

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 133237



Operator Signature  
ANNA NAREHOOD

Remarks: *Diagnostic*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      08:14

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:15
02 Std. Sol.	0.000	08:16
03 Room Air	0.000	08:16
04 Std. Sol.	0.000	08:17
05 Room Air	0.000	08:17
06 Std. Sol.	0.000	08:18
07 Room Air	0.000	08:18

08 Sim Temp = 34.0°C

Simul Ser No = MP5321  
Std Sol No = NA      Oper No. = 133237  
County = 08

  
\_\_\_\_\_  
Operator Signature  
ANNA NAREHOOD

Remarks: Zero AC Check

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      08:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:20
02 Std. Sol.	0.022	08:21
03 Room Air	0.000	08:22
04 Std. Sol.	0.022	08:22
05 Room Air	0.000	08:23
06 Std. Sol.	0.022	08:24
07 Room Air	0.000	08:24

08 Sim Temp = 34.0°C

Simul Ser No = MP3002  
Std Sol No = 202301E      Oper No. = 133237  
County = 08

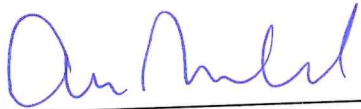  
\_\_\_\_\_  
Operator Signature  
ANNA NAREHOOD

Remarks: *Low Ac check - 0.020Ac*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      08:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:26
02 Std. Sol.	0.385	08:27
03 Room Air	0.000	08:27
04 Std. Sol.	0.388	08:28
05 Room Air	0.000	08:29
06 Std. Sol.	0.388	08:29
07 Room Air	0.000	08:30


08 Sim Temp = 34.0°C

Simul Ser No = MP6036

Std Sol No = 202212C

County = 08

Oper No. = 133237

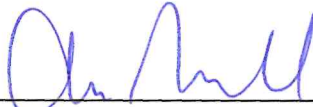

Operator Signature  
ANNA NAREHOOD

Remarks: High Ac Check - 0.400Ac

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005939  
Location = TOXL                    8164.14.00 09/16  
04/17/2024                            08:33

Test	AC	Time
01 Room Air	0.000	08:34
02 *Subject Test	INT*	08:35
03 Room Air	0.000	08:35

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = OTH                    Cit = INTERFERENT ICS  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08                    Oper No. = 133237

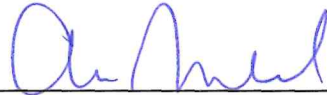

Operator Signature  
ANNA NAREHOOD

Remarks: Interferent check  
0.10% ethanol + 0.05% acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      08:42

Test	AC	Time
01 Room Air	0.000	08:43
02 *Subject Test	INT*	08:44
03 Room Air	0.000	08:44

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = OTH      Cit = INTERFERENT ACE  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08      Oper No. = 133237



Operator Signature  
ANNA NAREHOOD

Remarks: *Interferent check*  
*0.05% acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005939  
Location = TOXL                    8164.14.00 09/16  
04/17/2024                            08:45

Test	AC	Time
01 Room Air	0.000	08:46
02 *Subject Test	INT*	08:47
03 Room Air	0.000	08:47

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = OTH                            Cit = INTERFERENT MET  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08                            Oper No. = 133237



Operator Signature  
ANNA NAREHOOD

Remarks: Interferent check  
0.10% Methanol

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      08:48

Test	AC	Time
01 Room Air	0.000	08:49
02 *Subject Test	INT*	08:49
03 Room Air	0.000	08:50

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = OTH      Cit = INTERFERENT ISO  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08      Oper No. = 133237



Operator Signature  
ANNA NAREHOOD

Remarks: Interferent check  
0.10% isopropanol

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-005939  
Location = TOXL                                      8164.14.00 09/16  
04/17/2024    08:50

Test	AC	Time
01 Room Air	AMB*	08:51
02 Room Air	0.000	08:52

\*Invalid Test  
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                                      Weight = 150  
Test = OTH    Cit = AMBIENT  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08    Oper No. = 133237

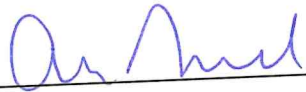  
\_\_\_\_\_  
Operator Signature  
ANNA NAREHOOD

Remarks: ambient fail

Form 106-18000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      08:54

Test	AC	Time
01 Room Air	0.000	08:55
02 Invalid Sample	X.XXX	08:55
03 Room Air	0.000	08:56

\*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = OTH      Cit = INVALID  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08      Oper No. = 133237



Operator Signature  
ANNA NAREHOOD

Remarks: Invalid sample

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-005939  
Location = TOXL                              8164.14.00 09/16  
04/17/2024    08:56

Test	AC	Time
01 Room Air	0.000	08:57
02 *Subject Test	ABT*	08:57
03 Room Air	0.000	08:58

\*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                              Weight = 150  
Test = OTH                              Cit = SEQUENCE ABORTE  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08                              Oper No. = 133237



Operator Signature  
ANNA NAREHOOD

Remarks: *Sequence aborted*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005939  
Location = TOXL                      8164.14.00 09/16  
04/17/2024                              09:09

Test	AC	Time
01 Room Air	0.000	09:10
02 Subject Test 1	0.000	09:10
03 Room Air	0.000	09:10
04 Reported AC	0.000	09:10

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                      Weight = 150  
Test = OTH                              Cit = ABA  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08                              Oper No. = 133237



Operator Signature  
ANNA NAREHOOD

Remarks: ABA test

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005939  
Location = TOXL                    8164.14.00 09/16  
04/17/2024                            09:11

Test	AC	Time
01 Diagnostic	OK	09:12
02 Room Air	0.000	09:12
03 Subject Test 1	0.000	09:12
04 Room Air	0.000	09:15
05 Std. Gas	0.035*	09:16
06 Room Air	0.000	09:16

\*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = OTH                            Cit = CAL CHECK OOT  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08                            Oper No. = 133237

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

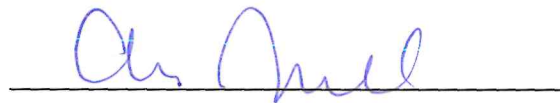

Operator Signature  
ANNA NAREHOOD

Remarks: Calibration check  
out of tolerance

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      09:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	09:17
02 Room Air	0.000	09:17

\*Invalid Test  
Inhibited - RFI

Lot No = 01923080A3  
Cyl No = 17  
Exp Date = 02/05/2025  
County = 08      Oper No. = 133237



Operator Signature  
ANNA NAREHOOD

Remarks: RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-005939  
Location = TOXL                                      8164.14.00 09/16  
04/17/2024    09:18


Test	AC	Time
01 Diagnostic	OK	09:18
02 Room Air	0.000	09:19
03 *Subject Test	0.000*	09:22
04 Room Air	0.000	09:26
05 Std. Gas	0.080	09:27
06 Room Air	0.000	09:29
07 *Subject Test	0.000*	09:32
08 Room Air	0.000	09:33
No RFI Detected		

\*Deficient Sample - Value Printed was  
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                                      Weight = 150  
Test = OTH    Cit = DEFICIENT  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08    Oper No. = 133237

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.

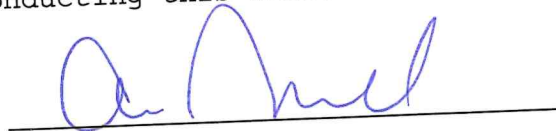

Operator Signature  
ANNA NAREHOOD

Remarks: Deficient Sample

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005939  
Location = TOXL                 8164.14.00 09/16  
04/17/2024                        09:33

Test	AC	Time
01 Diagnostic	OK	09:34
02 Room Air	IPS*	09:34
03 Room Air	0.000	09:35

\*Improper Sample

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                 Weight = 150  
Test = OTH                         Cit = INPROPER  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08                         Oper No. = 133237

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

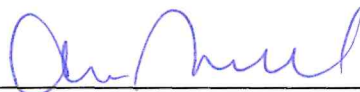

Operator Signature  
ANNA NAREHOOD

Remarks: Improper Sample

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      09:37

Test	AC	Time
01 Diagnostic	OK	09:38
02 Room Air	0.000	09:39
03 Subject Test 1	0.104*	09:39
04 Room Air	0.000	09:41
05 Std. Gas	0.079	09:43
06 Room Air	0.000	09:44
07 Subject Test 2	0.000*	09:44
08 Room Air	0.000	09:45

\*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = OTH      Cit = DIFF TO GREAT  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08      Oper No. = 133237

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

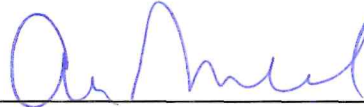

Operator Signature  
ANNA NAREHOOD

Remarks: *Difference Too Great*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-005939  
Location = TOXL                                      8164.14.00 09/16  
04/17/2024    09:46


Test	AC	Time
01 Diagnostic	OK	09:46
02 Room Air	0.000	09:47
03 Subject Test 1	0.000	09:47
04 Room Air	0.000	09:50
05 Std. Gas	0.077	09:51
06 Room Air	0.000	09:52
07 Subject Test 2	0.000	09:53
08 Room Air	0.000	09:54
09 Reported AC	0.000	09:47

Difference OK  
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                                      Weight = 150  
Test = OTH    Cit = CMS  
Dr. Lic. = ND/DIS821456  
Lot No = 01923080A3  
Cyl No = 17  
Expiration Date = 02/05/2025  
County = 08                                      Oper No. = 133237

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

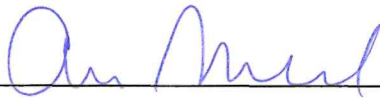

Operator Signature  
ANNA NAREHOOD

Remarks: CMS test

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005939  
Location = TOXL      8164.14.00 09/16  
04/17/2024      09:55

Test	AC	Time
01 Room Air	0.000	09:56
02 Std. Gas	0.079	09:56
03 Room Air	0.000	09:57
04 Std. Gas	0.080	09:57
05 Room Air	0.000	09:58
06 Std. Gas	0.081	09:58
07 Room Air	0.000	09:58
08 Std. Gas	0.080	09:59
09 Room Air	0.000	09:59
10 Std. Gas	0.080	10:00
11 Room Air	0.000	10:00
12 Std. Gas	0.080	10:01
13 Room Air	0.000	10:01
14 Std. Gas	0.080	10:01
15 Room Air	0.000	10:02
16 Std. Gas	0.081	10:02
17 Room Air	0.000	10:03
18 Std. Gas	0.080	10:03
19 Room Air	0.000	10:04
20 Std. Gas	0.081	10:04
21 Room Air	0.000	10:05
22 Std. Gas	0.080	10:05
23 Room Air	0.000	10:05
24 Std. Gas	0.080	10:06
25 Room Air	0.000	10:06
26 Std. Gas	0.081	10:07
27 Room Air	0.000	10:07
28 Std. Gas	0.081	10:08
29 Room Air	0.000	10:08
30 Std. Gas	0.081	10:08
31 Room Air	0.000	10:09
32 Std. Gas	0.081	10:09
33 Room Air	0.000	10:10
34 Std. Gas	0.081	10:10
35 Room Air	0.000	10:11
36 Std. Gas	0.081	10:11
37 Room Air	0.000	10:12
38 Std. Gas	0.081	10:12
39 Room Air	0.000	10:13
40 Std. Gas	0.080	10:13
41 Room Air	0.000	10:13
42 Cal Check Stats:		
43 Average	0.0805	
44 Std Dev	0.0006	
45 Rel Std Dev	0.76%	



Operator Signature  
ANNA NAREHOOD

Remarks:

Stability Test

Form 106-I8000