



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-004950	Instrument Location BMD
Reason for Install/Repair	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) Annual Inspection at Crime Laboratory	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date 4/19/24
---	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>[Signature]</i>	Certified Date 29 April 2024
---	--

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = BMDC 8164.14.00 09/16
04/19/2024 14:46

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 28423080A3
Standard Cyl #? 6
Standard Expiration? 11/05/2025
Oper No? 131854

Flow Cal. Date: 01/18/2010
Slope 774
Intercept -908904

IR Calibration Date: 04/19/2024
 3um 9um

0th Coef(*100):	-28189	-35353
1st Coef(*100):	278200	142117
2nd Coef(*100):	2249	1342
H2O adj(mg/l*10k):	369	463

***** Printer Test End *****



Operator Signature
NICK SCHOEDEL

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = BMDC 8164.14.00 09/16
04/19/2024 14:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:47
02 Std. Gas	0.081	14:48
03 Room Air	0.000	14:48
04 Std. Gas	0.080	14:48
05 Room Air	0.000	14:49
06 Std. Gas	0.080	14:49
07 Room Air	0.000	14:50

Lot No = 28423080A3
Cyl No = 6
Exp Date = 11/05/2025
County = 08 Oper No. = 131854



Operator Signature
NICK SCHOEDEL

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004950
Location = BMDC 8164.14.00 09/16
04/19/2024 14:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	14:54
02 Room Air	RFI*	14:54

*Invalid Test
Inhibited - RFI

Lot No = 28423080A3
Cyl No = 6
Exp Date = 11/05/2025
County = 08 Oper No. = 131854



Operator Signature
NICK SCHOEDEL

Remarks: *RFI Test*

Form 106-I8000