



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

| | |
|--|---|
| Serial Number 80-007097 | Instrument Location Burke County Sheriff's Shop |
| Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) Annual Inspection at Crime Laboratory | |

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

| | |
|-------------------------------|------------------------|
| Field Inspector Signature | Date 4/17/24 |
|-------------------------------|------------------------|

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

| | |
|---------------------------|------------------------------------|
| Reviewed/Certified By | Certified Date 24May2024 |
|---------------------------|------------------------------------|

When the paperwork was received, #3-6 were not checked off. Spoke to Chris Amlie and he confirmed that the Breath tube was heated as well as the datetime was correct and the gas cylinder was correct. AEN 24May24

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = BURK 8164.16.00 09/18
04/17/2024 12:11

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 14323080A4
Standard Cyl #? 41
Standard Expiration? 06/05/2025
Oper No? 120804

Flow Cal. Date: 09/03/2020
Slope 660
Intercept -636264

IR Calibration Date: 04/04/2024
 3um 9um

0th Coef(*100): -25634 -25706
1st Coef(*100): 238131 134910
2nd Coef(*100): 3596 1460
H2O adj(mg/l*10k): 614 521

***** Printer Test End *****

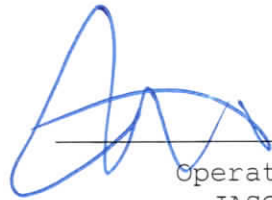

Operator Signature
JASON CARTIER

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = BURK 8164.16.00 09/18
04/17/2024 12:12

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 12:12 |
| 02 Std. Gas | 0.079 | 12:13 |
| 03 Room Air | 0.000 | 12:13 |
| 04 Std. Gas | 0.079 | 12:14 |
| 05 Room Air | 0.000 | 12:14 |
| 06 Std. Gas | 0.079 | 12:14 |
| 07 Room Air | 0.000 | 12:15 |

Lot No = 14323080A4
Cyl No = 41
Exp Date = 06/05/2025
County = 07 Oper No. = 120804



Operator Signature
JASON CARTIER

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007097
Location = BURK 8164.16.00 09/18
04/17/2024 14:03

DRY CAL CHECK

| Test | AC | Time |
|-------------|------|-------|
| 01 Room Air | RFI* | 14:03 |
| 02 Room Air | RFI* | 14:03 |

*Invalid Test
Inhibited - RFI

Lot No = 14323080A4
Cyl No = 41
Exp Date = 06/05/2025
County = 07 Oper No. = 120804



Operator Signature
JASON CARTIER

Remarks:

Form 106-I8000