



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number	80-007086	Instrument Location	Lisbon, ND PD
Reason for Install/Repair			
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) Annual Inspection at Crime Laboratory			

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature	Jorge Gonzalez RC SO 4551	Date	06-21-24
---------------------------	---------------------------	------	----------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By	Ame Nahl	Certified Date	24 June 2024
-----------------------	----------	----------------	--------------



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = LISB      8164.16.00 09/18  
06/21/2024      18:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:19
02 Std. Gas	0.082	18:19
03 Room Air	0.000	18:20
04 Std. Gas	0.082	18:20
05 Room Air	0.000	18:21
06 Std. Gas	0.081	18:21
07 Room Air	0.000	18:22

Lot No = 14323080A4  
Cyl No = 9  
Exp Date = 06/05/2025  
County = 37      Oper No. = 031021

  
Operator Signature  
JORGE GONZALEZ

Remarks:

Form 106-I8000

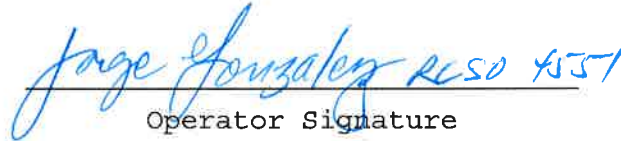
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = LISB      8164.16.00 09/18  
06/21/2024      18:23

Test	AC	Time
01 Room Air	RFI*	18:26
02 Room Air	0.000	18:27

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST TEST  
Sub DOB = 01/01/1991  
Sub Sex = Male      Weight = 180  
Test = OTH      Cit = 000000  
Dr. Lic. = ND/NA  
Lot No = 14323080A4  
Cyl No = 9  
Expiration Date = 06/05/2025  
County = 37      Oper No. = 031021

  
Operator Signature  
JORGE GONZALEZ

Remarks:

Form 106-I8000