



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number <u>80-006678</u>	Instrument Location <u>GFPD</u>
Reason for Install/Repair	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Annual Inspection at Crime Laboratory</u>	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): -annual inspection at crime laboratory
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <u><i>[Signature]</i></u>	Date <u>6-22-2024</u>
---	-----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <u><i>[Signature]</i></u>	Certified Date <u>24 June 2024</u>
---	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = GFPD 8164.14.00 09/16
06/22/2024 22:31

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 01923080A3
Standard Cyl #? 57
Standard Expiration? 02/05/2025
Oper No? 133510

Flow Cal. Date: 08/22/2016
Slope 686
Intercept -611596

IR Calibration Date: 08/22/2016
 3um 9um

0th Coef(*100): -15445 -21650
1st Coef(*100): 251854 131766
2nd Coef(*100): 3094 1378
H2O adj(mg/l*10k): 549 513

***** Printer Test End *****



Operator Signature
ADAM COLE

Remarks: *instal*

Form 106-I8000

I CERTIFY THAT THIS DOCUMENT IS
A TRUE AND CORRECT DUPLICATE OF
THE ORIGINAL

NAME *Adam Cole*
DATE 6-22-24
Police Dept. Grand Forks, ND
AEN


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = GFPD 8164.14.00 09/16
06/22/2024 22:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	22:32
02 Std. Gas	0.081	22:33
03 Room Air	0.000	22:33
04 Std. Gas	0.081	22:33
05 Room Air	0.000	22:34
06 Std. Gas	0.080	22:34
07 Room Air	0.000	22:35

Lot No = 01923080A3
Cyl No = 57
Exp Date = 02/05/2025
County = 18 Oper No. = 133510

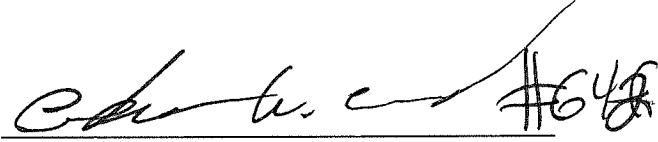

Operator Signature
ADAM COLE

Remarks: *Instal*

Form 106-I8000

I CERTIFY THAT THIS DOCUMENT IS
A TRUE AND CORRECT DUPLICATE OF
THE ORIGINAL
NAME *Adam Cole*
DATE *6-22-24*
Police Dept, Grand Forks, ND

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = GFPD 8164.14.00 09/16
06/22/2024 22:36

Test	AC	Time
01 Room Air	0.000	22:37
02 *Subject Test	RFI*	22:37
03 Room Air	RFI*	22:38

*Invalid Test
Inhibited - RFI

Sub Name = TEST, INSTAL NA
Sub DOB = 01/01/2000
Sub Sex = Male Weight = 150
Test = DUI Cit = 1
Dr. Lic. = ND/NA
Lot No = 01923080A3
Cyl No = 57
Expiration Date = 02/05/2025
County = 18 Oper No. = 133510



Operator Signature
ADAM COLE

Remarks: *Instal*

Form 106-I8000

I CERTIFY THAT THIS DOCUMENT IS
A TRUE AND CORRECT DUPLICATE OF
THE ORIGINAL

NAME *Adam Cole*

DATE *6-22-24*

Police Dept, Grand Forks, ND