



NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00 3061

A. General Setup and Checks:

1. Initial start up
 - Power On
 - 5-minute countdown during initial warm-up
 - 1-minute countdown to begin test after standby mode
2. Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
 - Level 1 (Operator)
 - Level 2 (Field Inspector)
 - Level 3 (Lab Personnel)
4. Printer Setup (Level 3,S,P)
5. Forms Load Completed (CMI, Inc., CoBrA Software)

B. Level 1 Setup and Checks:

1. Internal Printer Test (Level 1,P). Sign and attach record.
 - Paper Feed
2. External Printer Test (Level 1,P). Sign and attach record.
3. Barcode Reader (Level 1,B) (Tone sounds after each)
 - Operator ID
 - Invalid ID
 - ND License
 - R key: Refusal. Sign and attach record.

C. Level 2 Setup and Checks:

1. Continuous Air Blank (Level 2,A)
2. Diagnostic (Level 2,D). Sign and attach test record.
3. Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
 - Inst Ser#: 80-003061 DSP Ver: 1362.14
 - Core Ver: 8164.14 CPLD Ver: 7

D. Level 3 Setup and Checks:

1. DVM Monitor (Level 3,D,D):

3: 13031 s: 1 C: 47.00
9: 13222 s: 1 B: 45

DVM (3 & 9) between 9,000-16,000 counts

RFI indicator with keyed radio: 071

2. Function (F) Keys:

F5 – Tone

F9 - Fan

F6 – Green LED

F10 – Pump

F7 – Red LED

F11 – Solenoid

3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 934 CF: 1.08

4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 86 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T):

C: 47.0 A: 46.5 I: 0.0

B: 45.0 V: 0.0

Cell Temperature (C) between 47 ± 0.02 °C

Breath Hose Temperature (B) between 35.5 – 47 °C

Internal Ambient Temperature (A) above 41 °C

6. Tank Monitor (Level 3,D,G):

Display: 830 psi Regulator: 800 psi

Display and Regulator ± 50 psi of each other Yes or No

Gas tank tare necessary? Yes or No

If Yes, display readings after tare (Level 3,M,C,G):

Display: _____ psi Regulator: _____ psi

7. General Setup (Level 3,S,G)

8. Configure Start Test (Level 3,S,T)

9. Network (Level 3,S,N)

E. Tests (Sign and attach all test records):

1. Configure wet bath simulator for the following test (Level 1,S)

Zero AC-Wet Calibration Check (Level 1,C):

Sim ser #: MP5321 Lot #: NIA Exp date: NIA

Known value: 0.000 AC

Results are Zero AC

2. Configure wet bath simulator for the following test (Level 1,S)

Low AC-Wet Calibration Check (Level 1,C):

Known Value ≤ 0.03 AC: 0.020 AC

Sim ser #: MP3002 Lot #: 202301E Exp date: 1/20/25

Results ± 0.005 of known AC

3. Configure wet bath simulator for the following test (Level 1,S)
 High AC-Wet Calibration Check (Level 1,C):
 Known Value \geq 0.25 AC: 0.400 AC
 Sim ser #: MP6036 Lot #: 202212C Exp date: 08DEC24
 Results \pm 5% of known AC
4. Configure dry gas standard for the following tests (Level 1,S)
 Known Value: 0.080 AC
 Gas Cylinder Lot #: 28423080A2 Cylinder #: 37 Exp. Date: 11/5/25
5. 0.10% Ethanol + 0.05% Acetone (Level 1,B):
 Sim. Ser #: DR7344 Lot #: ICS 8 Exp. Date: NIA
 Display reads "Interferent Detect"
6. 0.05% Acetone (Level 1,B):
 Sim. Ser #: DR7111 Lot #: ACE3 Exp. Date: NIA
 Display reads "Interferent Detect"
7. 0.10% Methanol (Level 1,B):
 Sim. Ser #: DR7345 Lot #: MET3 Exp. Date: NIA
 Display Reads "Interferent Detect"
8. 0.10% Isopropanol (Level 1,B):
 Sim. Ser #: DR7350 Lot #: ISO3 Exp. Date: NIA
 Display Reads "Interferent Detect"
9. Ambient Fail (Level 1,B): Display read and test record correct.
10. Invalid Sample (Level 1,B): Display read and test record correct.
11. Sequence Aborted (Level 1,B): Display read test record correct.
12. ABA (Level 1,B): Test record correct.
13. Calibration check out of tolerance (CMS mode): Display read and test record correct.
14. RFI Detected (B,C, or CMS mode): Display read and test record correct.
15. Tank Pressure Below Minimum (CMS mode): Display reads correct.
16. Deficient Samples (CMS mode): Display read and test record correct.
17. Improper Sample (CMS mode): Display read and test record correct.
18. Difference Too Great (CMS mode): Display read and test record correct.
19. CMS (CMS mode): Test record correct.
20. Stability Test (Level 3,T,S):
 Ave: 0.0794 Std Dev: 0.0005 Rel Std Dev: 0.6476

Maintenance/Repairs/Notes: NIA

Instrument is acceptable to be used in the field. Yes or No
If No, state reason(s) why:



Inspector Signature

24 June 2024

Date



Reviewer

25 June 2024

Date

Intoxilyzer Test Record and Checklist
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 08:42

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMN

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

24 characters per line
abcdefghijklmnopqrstuvw
ABCDEFGHIJKLMNQRSTUW

32 characters per line
abcdefghijklmnopqrstuvwxy123456
ABCDEFGHIJKLMNQRSTUWXYZ123456

42 characters per line
abcdefghijklmnopqrstuvwxy1234567890abcdef
ABCDEFGHIJKLMNQRSTUWXYZ1234567890ABCDEF

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Menfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Wet
Standard Value? 0.020
Standard Solution #: 202301E
Sim Serial #: MP3002
Oper No? 133237

Flow Cal. Date: 03/04/2011
Slope: 700
Intercept: -623726

IR Calibration Date: 04/10/2015
3um 9um

0th Coef(*100):	-25722	-17879
1st Coef(*100):	266163	136703
2nd Coef(*100):	2302	1151
H2O adj(mg/l*10K):	412	294

***** Test End *****



Operator Signature
ANNA NAREHOOD

Remarks: *Internal
print test*

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 09:27

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Wet
Standard Value? 0.020
Standard Solution #? 202301E
Sim Serial #? MP3002
Oper No? 133237

Flow Cal. Date: 03/04/2011
Slope 700
Intercept -623726

IR Calibration Date: 04/10/2015
 3um 9um

0th Coef(*100): -25722 -17879
1st Coef(*100): 266163 136703
2nd Coef(*100): 2302 1151
H2O adj(mg/l*10k): 412 294

***** Printer Test End *****

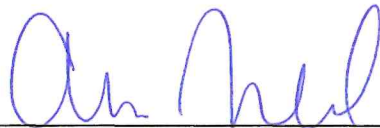

Operator Signature
ANNA NAREHOOD

Remarks: External Print Test

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 09:27

Test	AC	Time
01 Room Air	0.000	09:28
02 *Subject Test	REF*	09:29
03 Room Air	0.000	09:29

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = REFUSAL
Dr. Lic. = ND/DIS821456
Simul Ser No = MP3002
Std Sol No = 202301E
County = 08 Oper No. = 133237

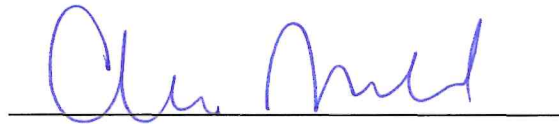

Operator Signature
ANNA NAREHOOD

Remarks: Refusal

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 09:31

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 133237

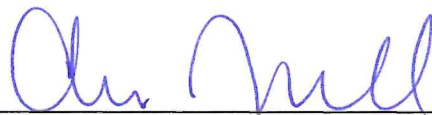

Operator Signature
ANNA NAREHOOD

Remarks: *Diagnostic*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 09:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:43
02 Std. Sol.	0.000	09:44
03 Room Air	0.000	09:45
04 Std. Sol.	0.000	09:45
05 Room Air	0.000	09:46
06 Std. Sol.	0.000	09:47
07 Room Air	0.000	09:47

08 Sim Temp = 34.0°C

Simul Ser No = MP5321

Std Sol No = NA

County = 08

Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: zero AC check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 09:51

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:52
02 Std. Sol.	0.019	09:53
03 Room Air	0.000	09:53
04 Std. Sol.	0.020	09:54
05 Room Air	0.000	09:55
06 Std. Sol.	0.019	09:55
07 Room Air	0.000	09:56

08 Sim Temp = 34.0°C

Simul Ser No = MP3002
Std Sol No = 202301E
County = 08 Oper No. = 133237

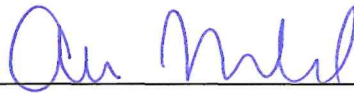

Operator Signature
ANNA NAREHOOD

Remarks: *LOW AC Check - 0.020Ac*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 09:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:58
02 Std. Sol.	0.393	09:59
03 Room Air	0.000	09:59
04 Std. Sol.	0.394	10:00
05 Room Air	0.000	10:00
06 Std. Sol.	0.394	10:01
07 Room Air	0.000	10:02
08 Sim Temp = 34.0°C		

Simul Ser No = MP6036
Std Sol No = 202212C
County = 08 Oper No. = 133237

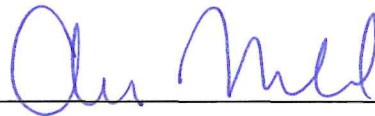

Operator Signature
ANNA NAREHOOD

Remarks: High Ae Check -0.400AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 10:07

Test	AC	Time
01 Room Air	0.000	10:08
02 *Subject Test	INT*	10:09
03 Room Air	0.000	10:10

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: *Interferent check*
0.10% ETOH + 0.05% Acetone

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 10:10

Test	AC	Time
01 Room Air	0.000	10:11
02 *Subject Test	INT*	10:12
03 Room Air	0.000	10:13

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INTERFERENT ACE
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

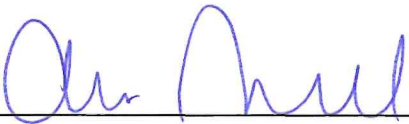

Operator Signature
ANNA NAREHOOD

Remarks: *Interferent check*
0.0590 Acetone

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 10:17

Test	AC	Time
01 Room Air	0.000	10:18
02 *Subject Test	INT*	10:20
03 Room Air	0.000	10:21

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INTERFERENT MET
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: Interferent check
0.10% MeOH

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 10:14

Test	AC	Time
01 Room Air	0.000	10:15
02 *Subject Test	INT*	10:15
03 Room Air	0.000	10:16

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INTERFERENT ISO
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

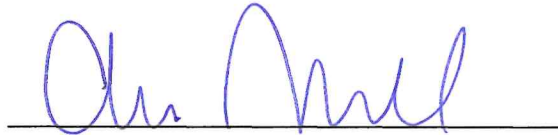

Operator Signature
ANNA NAREHOOD

Remarks: *interferent check*
0.10% Isopropanol

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 11:38

Test	AC	Time
01 Room Air	AMB*	11:38
02 Room Air	0.000	11:39

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = AMBIENT FAIL
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: *Ambient Fail*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 11:40

Test	AC	Time
01 Room Air	0.000	11:40
02 Invalid Sample	X.XXX	11:41
03 Room Air	0.000	11:41

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INVALID SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

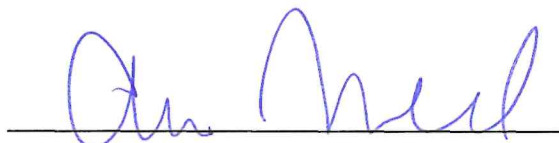

Operator Signature
ANNA NAREHOOD

Remarks: *Invalid Sample*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 11:42

Test	AC	Time
01 Room Air	0.000	11:43
02 *Subject Test	ABT*	11:43
03 Room Air	0.000	11:43

*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = SEQ ABORTED
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

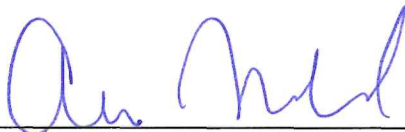

Operator Signature
ANNA NAREHOOD

Remarks: *Sequence aborted*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 10:55

Test	AC	Time
01 Room Air	0.000	10:56
02 Subject Test 1	0.000	10:56
03 Room Air	0.000	10:57
04 Reported AC	0.000	10:56

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = ABA TEST
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

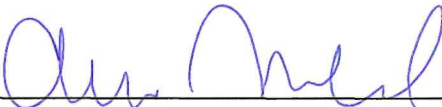

Operator Signature
ANNA NAREHOOD

Remarks: ABA Test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 11:44

Test	AC	Time
01 Diagnostic	OK	11:45
02 Room Air	0.000	11:45
03 Subject Test 1	0.000	11:46
04 Room Air	0.000	11:48
05 Std. Gas	0.040*	11:49
06 Room Air	0.000	11:49

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = CAL CHECK OOT
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

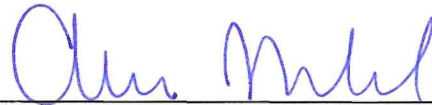

Operator Signature
ANNA NAREHOOD

Remarks: Calibration check out of tolerance.

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 11:50

Test	AC	Time
01 Diagnostic	OK	11:51
02 Room Air	RFI*	11:51
03 Room Air	0.000	11:52

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

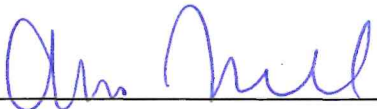

Operator Signature
ANNA NAREHOOD

Remarks: RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 11:53

Test	AC	Time
01 Diagnostic	OK	11:54
02 Room Air	0.000	11:54
03 *Subject Test	0.000*	11:57
04 Room Air	0.000	12:00
05 Std. Gas	0.020**	12:02
06 Room Air	0.000	12:02

*Deficient Sample - Value Printed was
Highest Obtained
**Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = DEFICIENT
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
ANNA NAREHOOD

Remarks: Deficient sample

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 12:03

Test	AC	Time
01 Diagnostic	OK	12:03
02 Room Air	IPS*	12:03
03 Room Air	0.000	12:04

*Improper Sample

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = IMPROPER SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

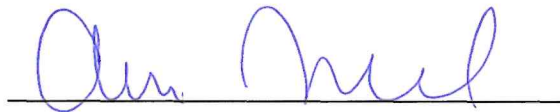

Operator Signature
ANNA NAREHOOD

Remarks: Improper Sample

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 12:19

Test	AC	Time
01 Diagnostic	OK	12:19
02 Room Air	0.000	12:20
03 Subject Test 1	0.388*	12:21
04 Room Air	0.000	12:23
05 Std. Gas	0.078	12:24
06 Room Air	0.000	12:26
07 Subject Test 2	0.000*	12:26
08 Room Air	0.000	12:27

*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = DIFF TOO GREAT
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

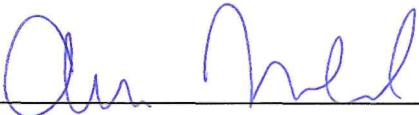

Operator Signature
ANNA NAREHOOD

Remarks: Difference Too Great

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 12:27


Test	AC	Time
01 Diagnostic	OK	12:28
02 Room Air	0.000	12:29
03 Subject Test 1	0.000	12:29
04 Room Air	0.000	12:31
05 Std. Gas	0.076	12:32
06 Room Air	0.000	12:33
07 Subject Test 2	0.000	12:34
08 Room Air	0.000	12:36
09 Reported AC	0.000	12:29

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = CMS TEST
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 37
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

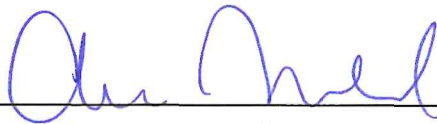

Operator Signature
ANNA NAREHOOD

Remarks: CMS Test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
06/24/2024 12:39

Test	AC	Time
01 Room Air	0.000	12:40
02 Std. Gas	0.079	12:40
03 Room Air	0.000	12:41
04 Std. Gas	0.079	12:41
05 Room Air	0.000	12:42
06 Std. Gas	0.079	12:42
07 Room Air	0.000	12:42
08 Std. Gas	0.079	12:43
09 Room Air	0.000	12:43
10 Std. Gas	0.080	12:44
11 Room Air	0.000	12:44
12 Std. Gas	0.079	12:45
13 Room Air	0.000	12:45
14 Std. Gas	0.080	12:45
15 Room Air	0.000	12:46
16 Std. Gas	0.080	12:46
17 Room Air	0.000	12:47
18 Std. Gas	0.080	12:47
19 Room Air	0.000	12:48
20 Std. Gas	0.079	12:48
21 Room Air	0.000	12:49
22 Std. Gas	0.080	12:49
23 Room Air	0.000	12:49
24 Std. Gas	0.080	12:50
25 Room Air	0.000	12:50
26 Std. Gas	0.079	12:51
27 Room Air	0.000	12:51
28 Std. Gas	0.080	12:51
29 Room Air	0.000	12:52
30 Std. Gas	0.079	12:52
31 Room Air	0.000	12:53
32 Std. Gas	0.079	12:53
33 Room Air	0.000	12:54
34 Std. Gas	0.079	12:54
35 Room Air	0.000	12:55
36 Std. Gas	0.080	12:55
37 Room Air	0.000	12:55
38 Std. Gas	0.079	12:56
39 Room Air	0.000	12:56
40 Std. Gas	0.079	12:57
41 Room Air	0.000	12:57
42 Cal Check Stats:		
43 Average	0.0794	
44 Std Dev	0.0005	
45 Rel Std Dev	0.64%	



Operator Signature
ANNA NAREHOOD

Remarks:

Stability Test

Form 106-I8000