



NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION

INTOXILYZER® 8000 REPAIR AND MAINTENANCE

Intoxilyzer® 8000 Serial Number: 80-00 5362 Maintenance Location: TOXL

1. Reported Issue and Issue Found:

Emails from McKenzie Co stating breath hose and instrument were not warming up and temperature regulation errors occurred.

2. Repair/Maintenance Performed:

Allowed instrument to warm up upon arrival. Turned instrument off to cool over a weekend. Turned on, ran diagnostics, let sit for 3 days and running diagnostic checks periodically.

3. Does the instrument flow sensor need to be adjusted? Yes or No

- If Yes, complete Flow Sensor Adjustment and Calibration (Document ID: 11872), then go to step 4.
- If No, go to step 4.

4. Does the instrument calibration need to be adjusted? Yes or No

- If Yes, complete Intoxilyzer® 8000 Calibration Adjustment (Document ID: 11859), then go to step 5.
- If No, go to step 5.

5. Was the instrument cover removed for the repair? Yes or No

- If Yes, complete Intoxilyzer® 8000 Initial Inspection (Document ID: 11784), then go to step 6.
- If No, go to step 6.

6. Is an Annual Inspection due on the Intoxilyzer® 8000? Yes or No


- If Yes, complete Intoxilyzer® 8000 Annual Inspection (Document ID: 11698), step 7 is not needed, and sign below.
- If No, go to step 7.

7. Was an Intoxilyzer® 8000 Initial Inspection (Document ID: 11784) completed in step 5? Yes or No
- If Yes, no further action is needed and sign below.
 - If No, Complete Intoxilyzer® 8000 Calibration (Document ID: 11871) and sign below.


Remarks/Notes: NIA

Instrument is acceptable to be used in the field. Yes or No

If No, state reason(s) why:

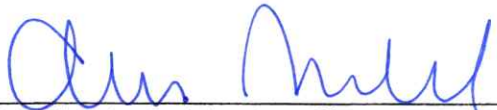
Breath Alcohol Analyst Signature

27 June 2024
Date


Reviewer Signature

27 June 2024
Date


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/24/2024 15:12

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 133237

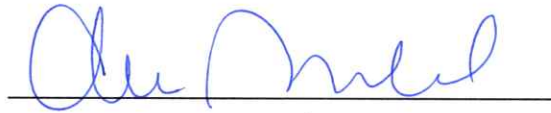

Operator Signature
ANNA NAREHOOD

Remarks: *Diagnostic Test*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/25/2024 07:41

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 133237

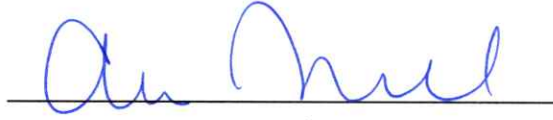

Operator Signature
ANNA NAREHOOD

Remarks: Diagnostic instrument on
overnight

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/27/2024 10:37

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 133237

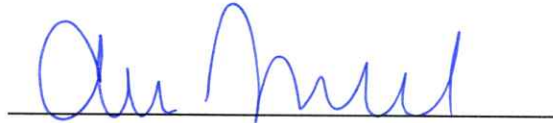

Operator Signature
ANNA NAREHOOD

Remarks: ^{After} Full day on standby
mode.

Form 106-I8000