



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-005943	Instrument Location NCSO - NELSON COUNTY SHERIFF'S OFFICE
Reason for Install/Repair	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 8/22/2024
-------------------------------	-------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Anna Narehood	Certified Date 03Sept2024
---------------------------	---------------	------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = NCSO 8164.14.00 09/16
08/22/2024 12:40

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

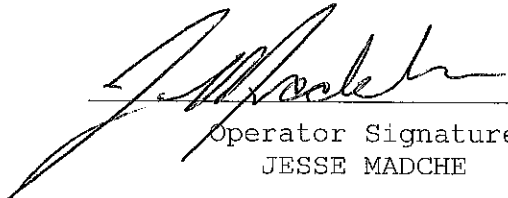
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 28423080A3
Standard Cyl #? 42
Standard Expiration? 11/05/2025
Oper No? 130917

Flow Cal. Date: 06/20/2013
Slope 686
Intercept -735246

IR Calibration Date: 08/26/2016

	3um	9um
0th Coef(*100):	-13589	-19547
1st Coef(*100):	284566	139614
2nd Coef(*100):	2717	1336
H2O adj(mg/l*10k):	381	404

***** Printer Test End *****


Operator Signature
JESSE MADCHE

Remarks:

Print test

Form 106-I8000

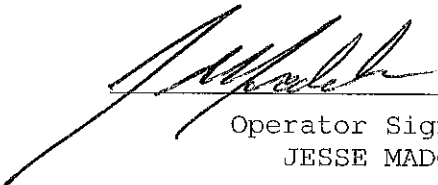
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = NCSO 8164.14.00 09/16
08/22/2024 12:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:43
02 Std. Gas	0.079	12:43
03 Room Air	0.000	12:44
04 Std. Gas	0.079	12:44
05 Room Air	0.000	12:45
06 Std. Gas	0.079	12:45
07 Room Air	0.000	12:45

Lot No = 28423080A3
Cyl No = 42
Exp Date = 11/05/2025
County = 32 Oper No. = 130917


Operator Signature
JESSE MADCHE

Remarks:

Cal check

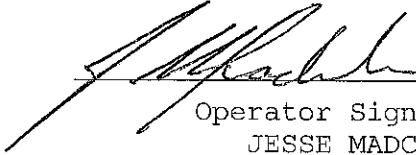
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005943
Location = NCSO 8164.14.00 09/16
08/22/2024 12:46

Test	AC	Time
01 Room Air	RFI*	12:47
02 Room Air	0.000	12:48

*Invalid Test
Inhibited - RFI


Operator Signature
JESSE MADCHE

Remarks:

RFI Test

Form 106-I8000